




Stick the Client Registration Number sticker onto the blood bag. Ensure the needle covers are securely in place, clamps closed and one knot tied in the tubing.




01

Place blood bag with absorbent cloth between the white gel pack and then both the blood bag and gel pack into clear bag and seal it.

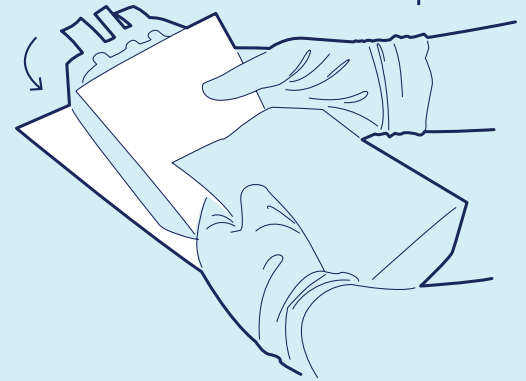

02

Place packaged blood bag into the silver foil bag and ensure that the checklist on foil bag sticker has been completed.

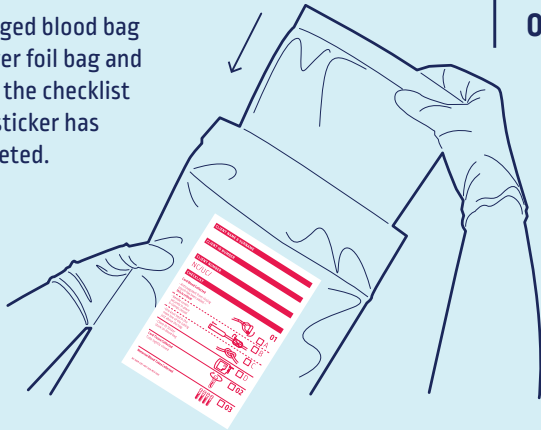

03

If cord tissue has been collected, the date and time of collection needs to be recorded on the yellow sticker.

Place the bag containing the cord tissue tube in the foil bag along with the packaged blood bag



04

After the maternal blood test samples have been drawn, place the 4 filled test tubes in the clear packet marked 'maternal blood collection'.

The date and time of collection needs to be recorded on the red sticker. Place this packet in the silver foil bag.



05


Place the foil bag into the collection kit box.

Please ensure that all information in the important folder is completed and placed into the collection kit box.



06

Please ensure that the box is then placed inside the plastic courier bag and the waybill is kept on the outside to give to the courier.

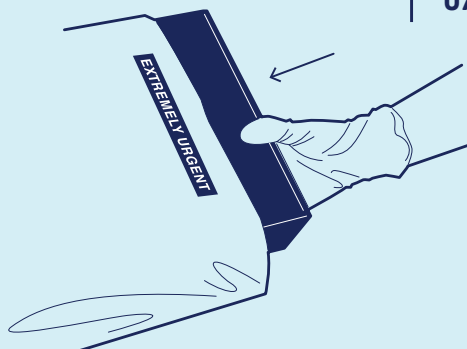

07

Please call the Client Services Team to arrange for the courier to collect the collection kit box or call

**084 664 4646**

Please note the cells do remain viable for 72 hours and we do endeavour to get them to our laboratory within 24 hours.



08