

VeriSeq NIPT Solution v2 Package Insert

FOR IN VITRO DIAGNOSTIC USE

Intended Use

The VeriSeq NIPT Solution v2 is an *in vitro* diagnostic test intended for use as a screening test for the detection of genome-wide fetal genetic anomalies from maternal peripheral whole blood specimens in pregnant women of at least 10 weeks gestation. VeriSeq NIPT Solution v2 uses whole genome sequencing to detect partial duplications and deletions for all autosomes and aneuploidy status for all chromosomes. The test offers an option to request the reporting of sex chromosome aneuploidy (SCA). This product must not be used as the sole basis for diagnosis or other pregnancy management decisions.

The VeriSeq NIPT Solution v2 includes: the VeriSeq NIPT Workflow Manager v2 for the VeriSeq NIPT Microlab STAR, the VeriSeq NIPT Sample Prep Kits, and the VeriSeq Onsite Server v2 with the VeriSeq NIPT Assay Software v2. The VeriSeq NIPT Solution v2 is intended to be used with a next generation sequencer.

Summary and Explanation of the Assay

Fetal chromosome abnormalities, specifically aneuploidy, which is an abnormal number of chromosomes, are a common cause of reproductive failure, congenital anomalies, developmental delay, and intellectual disabilities. Aneuploidy affects approximately 1 in 300 live births, with much higher rates associated with miscarriage and stillbirth.^{1,2} Until recently, there have been two types of prenatal tests for these disorders: diagnostic testing or screening. Diagnostic testing involves invasive procedures such as amniocentesis or chorionic villus sampling. These testing methods are considered the gold standard for detection of fetal aneuploidy. However, they are associated with a risk of pregnancy loss between 0.11% and 0.22%.³ Conventional multiple marker screens have no risk of pregnancy loss as they are noninvasive, but they are less accurate than diagnostic tests. Their detection rates for trisomy 21 vary between 69–96% depending on the particular screen, maternal age, and gestational age at testing.⁴ Importantly, they have false positive rates of approximately 5%, which can lead to invasive diagnostic testing for confirmation and, thus, the risk of procedure-related pregnancy loss.⁴ Ultrasound screens can also detect chromosome abnormalities, but they do so with even less certainty than these other methods.

Fetal aneuploidy for chromosomes 21, 18, 13, X and Y can be detected with a high degree of accuracy by noninvasive prenatal testing (NIPT) using whole-genome sequencing of cell-free DNA (cfDNA) obtained from maternal plasma at 10 weeks gestation or later. A recent meta-analysis of multiple clinical studies reported the weighted pooled detection rates and specificities for trisomy 21 and trisomy 18 in singleton pregnancies as follows: trisomy 21 99.7% and 99.96% and trisomy 18 97.9% and 99.96%, respectively.⁵ One study suggests that use of NIPT as a primary screen across all pregnancies could result in an 89% reduction in the number of confirmatory invasive procedures.⁶

Given the significant reduction of false positive rates with NIPT compared to conventional multiple marker screening, numerous professional medical organizations have issued opinion statements supporting several indications for the use of NIPT.

Specifically, the International Society for Prenatal Diagnosis, the American College of Obstetricians and Gynecologists (ACOG) /Society for Maternal Fetal Medicine (SMFM), the American College of Medical Genetics and Genomics (ACMG), and the European Society of Human Genetics/American Society of Human Genetics support offering NIPT to all pregnant women.^{7,8,9} Pretest counseling, informed consent, and diagnostic testing to confirm a positive cfDNA screening result are recommended.⁴

The VeriSeq NIPT Solution v2 is a noninvasive *in vitro* diagnostic (IVD) test that utilizes whole-genome sequencing of cfDNA fragments derived from maternal peripheral whole blood samples from pregnant women of at least 10 weeks gestation. The test offers two options for types of screening: basic and genomewide. The

basic screening provides information on the aneuploidy status of chromosomes 21, 18, 13, X, and Y only. Genomewide screenings provide partial duplications and deletions for all autosomes and aneuploidy status for all chromosomes. Both screening types provide the option for sex chromosome aneuploidy (SCA) reporting with or without fetal sex reporting. The reporting option for SCA can be turned off. If the reporting option for SCA is turned off, fetal sex is not reported either. For more information on the sex reporting options, refer to the *VeriSeq NIPT Solution v2 Software Guide (document # 1000000067940)*.

Principles of Procedure

The VeriSeq NIPT Solution v2 is an automated solution for laboratory NIPT testing which consists of automated sample preparation and sequencing data analysis. The VeriSeq NIPT Sample Prep Kits are specialized reagents that are used in conjunction with the VeriSeq NIPT Microlab STAR to prepare 24-, 48-, or 96-sample batches for next generation sequencing. Whole genome, paired-end sequencing data is analyzed by specialized software, the VeriSeq NIPT Assay Software v2, and a report is generated.

The workflow consists of the following procedures: sample collection, plasma isolation, cfDNA extraction, library preparation, library quantification, library pooling, sequencing, and analysis which are outlined in more detail:

- ▶ **Sample Collection**—7–10 ml of maternal peripheral whole blood is collected in a Streck cell-free DNA Blood Collection Tube (BCT), which prevents cell lysis and genomic contamination and stabilizes whole blood.
- ▶ **Plasma Isolation**—Within 5 days of collection, plasma is isolated from maternal peripheral whole blood using standard centrifugation techniques. The VeriSeq NIPT Microlab STAR aspirates and dispenses plasma into a 96-well deep-well plate for subsequent processing. In the event that retesting is required, post-processing samples may be recapped and stored at 4°C for an additional 5 days (up to a total of 10 days after blood collection).



CAUTION

Exceeding the aforementioned storage times can negatively impact individual sample failure rates.

- ▶ **cfDNA Extraction**—Purification of cfDNA from plasma is achieved by adsorption onto a binding plate, washing the binding plate to remove contaminants, and eluting.
- ▶ **Library Preparation**—The purified cfDNA fragments undergo an end repair process to convert 5' and 3' overhangs to blunt ends. Next, a deoxyadenosine nucleotide is added to the 3' ends to create a single base overhang. Indexed adapters containing a single base 3' deoxythymidine overhang are then ligated onto the processed cfDNA fragments. The ligated DNA is purified using solid phase reverse immobilization beads. Each sample in a set of 24, 48, or 96 receives a unique indexed adapter. The adapters serve 2 purposes:
 - ▶ Indexes allow sample identification in subsequent sequencing.
 - ▶ Index adapters contain sequences that allow for library capture on the solid surface of a sequencing flow cell for cluster generation and subsequent sequencing.
- ▶ **Quantification**—The library product is quantified using a fluorescent dye with concentration determined by comparison to a DNA standard curve.
- ▶ **Library Pooling and Sequencing**—The sample libraries are pooled together into 24- or 48-sample pools in adjusted amounts to minimize variation in coverage. Each pool is then sequenced using a next-generation sequencer.
- ▶ The VeriSeq NIPT Solution v2 does not include sequencing equipment and consumables.
- ▶ **Analysis**—For each sample, analysis consists of the following:
 - ▶ Identification of library fragments by index sequence and alignment of the paired end reads to a human reference genome.
 - ▶ Estimation of the fetal fraction of the library by combining information from the distribution of both the lengths and genomic coordinates of the library fragments.
 - ▶ After accounting for known biases, a statistical model detects regions of the genome which are under or overrepresented in the library in a manner consistent with an anomaly at the estimated level of fetal fraction.

- ▶ The NIPT report provides summary results for the selected test menu where ANOMALY DETECTED or NO ANOMALY DETECTED is listed along with a fetal fraction estimate for samples passing QC.
- ▶ The Supplementary Report provides quantitative metrics which characterize each detected anomaly. Use of the Supplementary Report is discretionary and not required.

Limitations of the Procedure

- ▶ The VeriSeq NIPT Solution v2 is a screening test and should not be considered in isolation from other clinical findings and test results. Conclusions about the fetal condition and pregnancy management decisions should not be based on the results of the NIPT screening alone.⁷
- ▶ The VeriSeq NIPT Solution v2 reports on the following:
 - ▶ Basic screen tests the overrepresentation of chromosomes 13, 18, and 21
 - ▶ Genomewide screen tests under and overrepresentation of all autosomes, including partial deletions and duplications of at least 7 Mb.
 - ▶ In singleton pregnancies with Yes or SCA selected as the sex reporting option, the following sex chromosomal anomalies: XO, XXX, XXY, and XYY.
 - ▶ In singleton pregnancies with Yes selected as the sex reporting option, fetal sex is reported.
 - ▶ The presence of a Y chromosome in twin pregnancies.
- ▶ Evidence supporting sensitivity and specificity for the test covers singleton and twin pregnancies. These instructions for use do not provide sensitivity or specificity data for triplets or higher-order pregnancies.
- ▶ The VeriSeq NIPT Solution v2 is not intended to detect polyploidy, such as triploidy.
- ▶ The VeriSeq NIPT Solution v2 is not intended to detect balanced chromosome rearrangements.
- ▶ The assay requires maternal peripheral whole blood samples from pregnant women of at least 10 weeks gestation.
- ▶ For basic screens, the VeriSeq NIPT Solution v2 test looks for specific chromosome abnormalities. Results reported as NO ANOMALY DETECTED do not eliminate the possibility of chromosomal abnormalities of the tested chromosomes. A negative result does not eliminate the possibility that the pregnancy has other chromosomal abnormalities, genetic conditions, or birth defects (eg, open neural tube defect).
- ▶ For genomewide screens, large deletions and duplications that are less than 75% of the size of the chromosome can be indicative of whole chromosome aneuploidy.
- ▶ For genomewide screens, certain regions are excluded from analysis. A list of such blacklisted regions is available on the Illumina Support website. Genomic anomaly detection is only performed on non-excluded regions.
- ▶ Fetal sex reporting is not available in all regions due to local regulations governing gender reporting.
- ▶ The results of the test can be confounded by certain maternal and fetal factors including but not limited to the following:
 - ▶ Recent maternal blood transfusion
 - ▶ Maternal organ transplant
 - ▶ Maternal surgical procedure
 - ▶ Maternal immunotherapy or stem cell therapy
 - ▶ Maternal malignancy
 - ▶ Maternal mosaicism
 - ▶ Fetoplacental mosaicism
 - ▶ Fetal demise
 - ▶ Nonviable twin

Product Components

The VeriSeq NIPT Solution v2 (part # 20030577) consists of the following:

- ▶ VeriSeq NIPT Sample Prep Kit (24 samples) (part # 20025895)
- ▶ VeriSeq NIPT Sample Prep Kit (48 samples) (part # 15066801)
- ▶ VeriSeq NIPT Sample Prep Kit (96 samples) (part # 15066802)
- ▶ VeriSeq Onsite Server v2 (part # 20028403) or an existing VeriSeq Onsite Server (part # 15076164 or # 20016240) that is upgraded to v2
 - ▶ VeriSeq NIPT Assay Software v2, preinstalled on the VeriSeq Onsite Server v2
- ▶ VeriSeq NIPT Microlab STAR (part# Hamilton Company Reno: 95475-01 (115V) & 95475-02 (230V), Hamilton Company Bonaduz: 806288)
 - ▶ VeriSeq NIPT Workflow Manager v2, preinstalled on the VeriSeq NIPT Microlab STAR

Reagents

Reagents Provided

Illumina provides the following reagents: VeriSeq NIPT Sample Prep Kit (24 samples) (part # 20025895), VeriSeq NIPT Sample Prep Kit (48 samples) (part # 15066801), and VeriSeq NIPT Sample Prep Kit (96 samples) (part # 15066802). The VeriSeq NIPT Sample Prep Kits are configured for use with the ML STAR (part # 95475-01, 95475-02, or 806288), which is provided by the Hamilton Company.

VeriSeq NIPT Sample Prep, Extraction Box

Table 1 VeriSeq NIPT Extraction Box (24) and (48), Part # 20025869 and 15066803

Reagent Name on Label	Number of Containers in Kit	Active Ingredients	Storage
Lysis Buffer	1	Guanidine hydrochloride in buffered aqueous solution	15°C to 30°C
Wash Buffer I	1	Guanidine hydrochloride and 2-propanol in buffered aqueous solution	15°C to 30°C
Wash Buffer II	1	Buffered aqueous solution containing salts	15°C to 30°C
Elution Buffer	1	Buffered aqueous solution	15°C to 30°C
Proteinase Buffer	1	Glycerol in buffered aqueous solution	15°C to 30°C
Proteinase K	3	Lyophilized Proteinase K	15°C to 30°C

Table 2 VeriSeq NIPT Extraction Box (96), Part # 15066807

Reagent Name on Label	Number of Containers in Kit	Active Ingredients	Storage
Lysis Buffer	1	Guanidine hydrochloride in buffered aqueous solution	15°C to 30°C
Wash Buffer I	1	Guanidine hydrochloride and 2-propanol in buffered aqueous solution	15°C to 30°C
Wash Buffer II	2	Buffered aqueous solution containing salts	15°C to 30°C
Elution Buffer	1	Buffered aqueous solution	15°C to 30°C

Reagent Name on Label	Number of Containers in Kit	Active Ingredients	Storage
Proteinase Buffer	1	Glycerol in buffered aqueous solution	15°C to 30°C
Proteinase K	4	Lyophilized Proteinase K	15°C to 30°C

VeriSeq NIPT Sample Prep, Library Prep Box

Table 3 VeriSeq NIPT Library Prep Box (24) and (48), Part # 20026030 and 15066809

Reagent Name on Label	Number of Containers in Kit	Active Ingredients	Storage
End Repair Mix	1	DNA polymerase and dNTPs in buffered aqueous solution	-25°C to -15°C
A-Tailing Mix	1	DNA polymerase and dATP in buffered aqueous solution	-25°C to -15°C
Ligation Mix	1	DNA ligase in buffered aqueous solution	-25°C to -15°C
Hybridization Buffer	1	Buffered aqueous solution	-25°C to -15°C
VeriSeq NIPT DNA Adapter Plate	1	Oligonucleotides in buffered aqueous solution	-25°C to -15°C

Table 4 VeriSeq NIPT Library Prep Box (96), Part # 15066810

Reagent Name on Label	Number of Containers in Kit	Active Ingredients	Storage
End Repair Mix	1	DNA polymerase and dNTPs in buffered aqueous solution	-25°C to -15°C
A-Tailing Mix	2	DNA polymerase and dATP in buffered aqueous solution	-25°C to -15°C
Ligation Mix	2	DNA ligase in buffered aqueous solution	-25°C to -15°C
Hybridization Buffer	1	Buffered aqueous solution	-25°C to -15°C
VeriSeq NIPT DNA Adapter Plate	1	Oligonucleotides in buffered aqueous solution	-25°C to -15°C

VeriSeq NIPT Sample Prep, Accessory Box

Table 5 VeriSeq NIPT Accessory Box, Part # 15066811

Reagent Name on Label	Number of Containers in Kit	Active Ingredients	Storage
DNA Binding Plate	1	Propylene microplate with modified silicone membrane	2°C to 8°C
Resuspension Buffer	1	Buffered aqueous solution	2°C to 8°C
Sample Purification Beads	1	Solid-phase paramagnetic beads in buffered aqueous solution	2°C to 8°C

Reagent Name on Label	Number of Containers in Kit	Active Ingredients	Storage
DNA Quantification Reagent	1	DNA intercalating dye in DMSO	2°C to 8°C
DNA Quantification Standard	1	dsDNA standard, non-specific DNA, and sodium azide in buffered aqueous solution	2°C to 8°C

VeriSeq NIPT Sample Prep, Workflow Tubes and Labels

Table 6 Workflow Tubes and Labels, Part # 15071543

Item Name on Label	Number of Items in Kit	Storage
Label (LBL)–Plate Barcode	9	15°C to 30°C
Label (LBL)–Deep-well Plate Barcode	12	15°C to 30°C
Tube (TB)–Empty Pooling Tube	5	15°C to 30°C

Reagents Not Provided

Reagents Required, Not Provided

- ▶ Sequencing reagents and consumables required for next-generation sequencing (NGS) system
- ▶ DNase/RNase-free water
- ▶ Ethanol, 100% (200 proof) of molecular biology grade



NOTE

Non-molecular biology grade ethanol can potentially negatively impact performance of the assay.

Reagents Optional, Not Provided

- ▶ Dulbecco's Phosphate-Buffered Saline (DPBS) for no template control (NTC)

Storage and Handling

- 1 Room temperature is defined as 15°C to 30°C.
- 2 All reagents are for one-time use only. After reagents are prepared for use, they should be used immediately.
- 3 If any of the packaging or contents of the VeriSeq NIPT Solution components are damaged or compromised, please contact Illumina Customer Service.
- 4 Reagents are stable when stored as indicated until the specified expiration date on the kit labels. For storage conditions, see the Storage column in the tables in *Reagents Provided on page 4*. Do not use expired reagents.
- 5 Changes in the physical appearance of the reagents provided can indicate deterioration of the materials. If changes in the physical appearance occur (eg, obvious changes in reagent color or cloudiness apparent with microbial contamination), do not use the reagents.
- 6 Adhere to the following best practices when handling Sample Purification Beads:
 - ▶ Never freeze the beads.
 - ▶ Allow the beads to reach room temperature before use.
 - ▶ Immediately before use, vortex the beads until well-suspended and color appears homogeneous.
- 7 Lysis Buffer, Wash Buffer I, Wash Buffer II, Elution Buffer, and Proteinase Buffer can form visible precipitates or crystals. Before use, vortex vigorously, and then visually inspect to make sure that no precipitates are present.
- 8 Never freeze whole blood after collection.
- 9 Sequence libraries as soon as possible after pooling. Pooled libraries are stable for up to 7 days at -25°C to -15°C. No additional denaturation is required if stored for this period of time at these conditions.

Equipment and Materials

Equipment and Materials Required, Not Provided

Equipment Required, Not Provided

Equipment	Supplier
20 µl single-channel pipettes	General lab supplier
200 µl single-channel pipettes	General lab supplier
1000 µl single-channel pipettes	General lab supplier
Pipette Aid	General lab supplier
Refrigerator, 2°C to 8°C	General lab supplier
Freezer, -25°C to -15°C	General lab supplier
Microcentrifuge	General lab supplier
Vortexer	General lab supplier
Centrifuge and rotor assembly for blood collection tubes	
Recommended:	
• Allegra X12R Series Centrifuge, 1600 g	Beckman Coulter, item #392304 (120 V or 230 V)
• Allegra Centrifuge GH-3.8 Rotor with buckets	Beckman Coulter, item #369704
• Allegra Centrifuge Bucket Covers, set of two	Beckman Coulter, item #392805
• Allegra Centrifuge Adapter Assembly, 16 mm, set of four	Beckman Coulter, item #359150
Equivalent:	General lab supplier
• Refrigerated centrifuge capable of 1600 × g with no-brake option	
• Swinging bucket rotor with buckets	
• Bucket inserts, 24, 48, or 96 tube capacity, 76 mm minimum depth	
• Insert adapters to support 16 x 100 mm blood collection tubes	
Centrifuge and rotor assembly for microplates	
Recommended:	
• Sorvall Legend XTR Centrifuge	Thermo Fisher Scientific, catalog # 75004521 (120 V) or catalog # 75004520 (230 V)
• HIGHPlate 6000 Microplate Rotor	Thermo Fisher Scientific, catalog # 75003606
• Two of either of the following support bases for microplates:	Thermo Fisher Scientific, catalog # 4379590
• MicroAmp 96-Well Support Base	Thermo Fisher Scientific, catalog # AB-0563/1000
• 96-Well PCR Plate Carrier	
Equivalent:	General lab supplier
• Centrifuge capable of 5600 × g	
• Swinging plate rotor with 96-well plate carriers, 76.5 mm minimum depth	
• Support base for microplates	
One of the following microplate readers (fluorometer) with SoftMax Pro v6.2.2 or higher:	
• Gemini XPS	Molecular Devices, part # XPS
• SpectraMax M2	Molecular Devices, part # M2
SpectraMax High-Speed USB, Serial Adapter	Molecular Devices, part # 9000-0938

Equipment	Supplier
Thermal cycler with the following specifications: <ul style="list-style-type: none"> • Heated lid • 4°C to 98°C temperature range • ±2°C temperature accuracy • 2°C per second minimum ramp rate • Compatible with Twin.tec PCR Plate 96-well, full skirt 	General lab supplier
VeriSeq NIPT Microlab STAR	Hamilton, part # 95475-01 (115 V), part # 95475-02 (230 V), or part # 806288 (for Hamilton Company Bonaduz)
A next-generation sequencing (NGS) system with the following capabilities: <ul style="list-style-type: none"> • 2 x 36 bp paired-end sequencing • Compatible with VeriSeq NIPT Sample Prep dual index adapters • Automatic production of .BCL files • Two channel chemistry • 400 million paired-end reads per run • Compatible with VeriSeq NIPT Assay Software v2 or a NextSeq 550Dx Sequencing System.	Instrument supplier or Illumina, part # 20005715
If using a NextSeq 550Dx Sequencing System: <ul style="list-style-type: none"> • NextSeq 550Dx High Output Reagent Kit v2.5, 75 cycles 	Illumina, part # 20028870
VeriSeq Onsite Server v2 or an upgraded VeriSeq Onsite Server	Illumina, part # 20028403 (v2) or #15076164 or # 20016240 (upgraded)

Optional Equipment, Not Provided

Equipment	Supplier
Pluggo Decapper System	LGP Consulting, part # 4600 4450
SpectraMax SpectraTest FL1 fluorescence validation plate	Molecular Devices, part # 0200-5060
Tube Revolver/Rotator, 15 ml tubes, 40 rpm, 100–240 V	Thermo Scientific, catalog # 88881001 (US) or catalog # 88881002 (EU)

Materials Required, Not Provided

Consumable	Supplier
1000 µl Conductive Non-Sterile Filter Tips	Hamilton, part # 235905
300 µl Conductive Non-Sterile Filter Tips	Hamilton, part # 235903
50 µl Conductive Non-Sterile Filter Tips	Hamilton, part # 235948
Deep-Well Reservoir	Corning Axygen, product # RES-SW96-HP-SI
MagNA Pure LC Medium Reagent Tub 20, 20 ml	Roche, product # 03004058001
Deep-Well Plate 96, 2 ml	Eppendorf, part # 0030505301
Low Volume 384 Well Black Flat Bottom Polystyrene Microplate	Corning, product # 3820
Twin.tec PCR Plate 96-well, full-skirt	Eppendorf, part # 0030129512
One of the following seals: <ul style="list-style-type: none"> • Microseal 'F' Foil • Foil seals 	Bio-Rad, catalog # MSF1001 Beckman Coulter, item # 538619
Cell-Free DNA BCT CE	Streck, catalog # 218997
Push Caps	Sarstedt, order # 65.802

Consumable	Supplier
2 ml Screw-cap tubes	General lab supplier
20 µl filter tips for 20 µl pipettor	General lab supplier
200 µl filter tips for 200 µl pipettor	General lab supplier
1000 µl filter tips for 1000 µl pipettor	General lab supplier
25 ml Serological Pipettes	General lab supplier
10 ml Serological Pipettes	General lab supplier
Recommended: Deconex® SOLARSEPT Deconex® 61 DR	Borer Chemie AG
Equivalent: An alcohol-based rapid disinfectant spray A solution of disinfecting detergent	General lab supplier

Optional Materials, Not Provided

Consumable	Supplier
Tube, screw cap, 10 ml (for control samples only)	Sarstedt, order # 60.551
Tube, screw cap, 50 ml	General lab supplier

Specimen Collection, Transport, and Storage



CAUTION

Handle all specimens as if they are potentially infectious agents.

- 1 Whole blood specimens of 7–10 ml must be collected in Streck Cell-Free DNA BCT.
- 2 Transportation of whole blood must comply with all applicable governing regulations for the transport of etiologic agents. Expedited shipping/transportation methods are recommended.
- 3 During transport, store at temperatures between 4°C and 30°C. After samples are received, store at 2°C to 8°C until ready to proceed. The time between blood collection and initial plasma isolation should not exceed 5 days.
- 4 In the event that retesting is required, post-processing samples may be recapped and stored at 4°C for an additional 5 days (up to a total of 10 days after blood collection).



CAUTION

Exceeding the aforementioned storage times can negatively impact individual sample failure rates.

Warnings and Precautions

- ▶ This assay contains Proteinase K. Personal injury can occur through inhalation, ingestion, skin contact, and eye contact. Use in a well-ventilated area, wear protective clothing, avoid breathing dust, and dispose of any containers and unused contents in accordance with applicable governmental safety standards.
- ▶ This assay contains guanidinium chloride. Personal injury can occur through inhalation, ingestion, skin contact, and eye contact. Use in a well-ventilated area, wear protective clothing, and dispose of any containers and unused contents in accordance with applicable local governmental safety standards.
- ▶ This assay contains 2-propanol, a flammable chemical. Keep away from heat and open flames. Personal injury can occur through inhalation, ingestion, skin contact, and eye contact. Use in a well-ventilated area, wear protective clothing, and dispose of any containers and unused contents in accordance with applicable local governmental safety standards.

- ▶ To prevent the formation of harmful gases, do not dispose of cfDNA extraction waste (contains guanidine thiocyanate) with waste containing bleach (sodium hypochlorite).
- ▶ Handle all specimens as if they contain potentially infectious agents.
- ▶ Use routine laboratory precautions. Do not pipette by mouth. Do not eat, drink, or smoke in designated work areas. Wear disposable gloves and laboratory coats when handling specimens and assay reagents. Wash hands thoroughly after handling specimens and assay reagents.
- ▶ Do not use any assay components beyond their stated expiration date on the assay box label. Do not interchange assay components from different assay lots. Assay lots are identified on the assay box label. Store the assay components at the specified temperature.
- ▶ To prevent sample or reagent degradation, make sure that all sodium hypochlorite vapors from cleaning have fully dissipated before starting the protocol.
- ▶ Failure to follow the procedures as outlined can result in erroneous results or significant reduction in sample quality.
- ▶ For environmental, health, and safety information, see the safety data sheets (SDS) at support.illumina.com/sds.html.

Procedural Notes

Avoiding Contamination

- ▶ Use fresh tips and fresh consumable labware.
- ▶ Use aerosol-resistant tips to reduce the risk of carry-over and sample-to-sample cross-contamination.
- ▶ Due to the potential for contamination, take extreme care to make sure that well contents remain fully in the well. Do not splash contents. Centrifuge following any vortexing step.
- ▶ Follow applicable governing regulations for proper laboratory practice and hygiene when handling blood and blood derivatives.
- ▶ Do not use aerosol bleach sprays when performing library preparation. Trace bleach contamination can lead to assay failure.

VeriSeq NIPT Microlab STAR Deck Cleaning

- ▶ Before use, inspect the deck for cleanliness. At least once a week, conduct the weekly maintenance and follow these cleaning instructions.
- ▶ Remove all unloadable carriers and clean with an alcohol-based rapid disinfectant spray (Deconex® SOLARSEPT or equivalent) and leave them to dry. If they are heavily soiled, soak them afterward in a solution of disinfecting detergent (Deconex® 61 DR cleaning liquid or equivalent), rinse with the alcohol-based disinfectant, and leave to dry.
- ▶ Open the front cover and wipe the deck with a cloth saturated with Deconex® SOLARSEPT (or equivalent). The slide blocks in particular must be checked for cleanliness.
- ▶ Remove the CVS manifold and clean manifold, gasket, and interior compartments of CVS with cloth.
- ▶ Empty tip waste for the CORE 96-head and the independent channel.
- ▶ Remove the independent channel tip eject plate of the tip waste station and clean it: spray Deconex® SOLARSEPT (or equivalent) directly onto the surface and wipe. Pull a new plastic bag over the frame and re-attach it. Put the clean tip eject plate back in place.
- ▶ Spray Deconex® SOLARSEPT (or equivalent) directly onto the surface of the CORE 96-head waste box and chute and wipe it clean.

- ▶ If buildup is difficult to remove from tip wastes, wipe with a cloth wet with DNase/RNase-free water until the buildup is removed. Dispose of the cloth appropriately. Proceed to sterilize with the alcohol-based disinfectant.
- ▶ Wet a lint-free cloth or cotton swab with 70% ethanol. Swab the laser scanner window of the barcode reader. Using the same cloth or swab, clean each well of the CPAC plate adapter. If using a cloth, press the cloth into each well of the adapter using the back of a pen to assure the interior of the well is properly cleaned.
- ▶ Clean the independent channels:
 - ▶ On the independent channels, clean the tip eject sleeve (outer part of the pipetting channels) with a lint-free cloth soaked in Deconex® SOLARSEPT (or equivalent). (See the *Hamilton Microlab STAR Reference Guide #15070074*.)
 - ▶ Clean the stop disk and the O-rings of the pipetting head (outer part of the pipetting channels) with a lint-free cloth soaked in Deconex® SOLARSEPT (or equivalent).
- ▶ Clean the CORE 96-head:
 - ▶ Using the same lint-free cloth soaked in Deconex® SOLARSEPT (or equivalent), clean the casing of the 96-head and the bottom of the stop disks.
 - ▶ Using the same cloth, or a torn strip of cloth soaked in Deconex® SOLARSEPT (or equivalent), ‘floss’ the cloth around the sides of the pipette channels of the 96-head in order to clean the o-rings. Repeat this procedure for every pipette channel on the 96-head.
- ▶ Spray the front and side cover with Deconex® SOLARSEPT (or equivalent), and wipe dry.
- ▶ Clean the Autoload protecting ribbon with a cloth soaked in Deconex® SOLARSEPT (or equivalent), and wipe without exerting pressure.
- ▶ When the deck and components are completely dry, replace the carriers.

**NOTE**

Improper cleaning and maintenance of the ML STAR can result in cross-contamination and poor assay performance.

Quality Control

Control material with known performance characteristics may be evaluated to detect differences in processing and technical procedures in the laboratory.

**NOTE**

Running a control sample or no template control reduces the total number of unknown maternal samples that can be processed with each sample prep kit.

Do not exceed two NTC samples per batch of 24 or 48 samples or four NTC samples per batch of 96 samples.

Instructions for Use

Tips and Techniques

Unless a safe stopping point is specified in the protocol, proceed immediately to the next step.

Barcoding Plates

- Barcodes for full-skirt plates start with PL.
- Barcodes for deep-well plates start with DW.
- Apply barcodes to full-skirt plates and deep-well plates on the side next to column 12.
- Load plates with the barcode facing right to enable automated scanning.

Sealing and Unsealing the Plate

- ▶ Always seal the 96-well plate before the following steps in the protocol:
 - ▶ Centrifuge steps
 - ▶ Thermal cycling steps
- ▶ To seal the plate, apply the adhesive cover to the plate and then seal.
- ▶ Before unsealing:
 - ▶ Briefly centrifuge the 96-well plate at 1000 × g for 20 seconds.
 - ▶ Place the plate on a flat surface before slowly removing the seal.

VeriSeq NIPT Microlab STAR

- ▶ Before use, perform and document the required maintenance according to the manufacturer instructions.
- ▶ Observe the ML STAR during the automated steps. Monitor the VeriSeq NIPT Workflow Manager v2 software interface for prompts and operator instructions.
- ▶ Keep the front cover in place during operation.
- ▶ Keep the deck clear of all items during operation.
- ▶ During plate vacuum steps, if prompted by the VeriSeq NIPT Workflow Manager v2, manually assist forming the seal between the plate and the vacuum manifold.
- ▶ Allow the system to dispose of tips from the adapter automatically. Do not manually remove tips unless prompted by the software.
- ▶ Remove spent reagents and used consumables as prompted by the Workflow Manager.
- ▶ Empty vacuum waste carboys daily. The first carboy should never exceed ½ full. Overflow of vacuum waste can damage the vacuum pump and can reduce the applied vacuum of the system.

Process Samples

Procedure

- 1 Complete the following steps for each aliquot:
 - a Centrifuge barcoded samples at 1600 × g for 10 minutes at 4°C with the brake off.
 - b When the centrifuge comes to a complete stop, remove the sample tubes.
After centrifugation, begin plasma isolation within 15 minutes. If more than 15 minutes elapse, centrifuge again.
- 2 Inspect each tube to confirm that it contains at least 1.5 ml plasma above the buffy coat.
- 3 Uncap the tubes and load them into the tube carriers. Load all samples and any plasma controls for the batch.

Isolate Plasma

Preparation

- 1 Label 1 deep-well plate Intermediate Plasma and apply a barcode.
- 2 Label 1 deep-well plate Final Plasma and apply a barcode.

Procedure

- 1 Open the AppLauncher, and then click VeriSeq NIPT Method.
- 2 Enter the Batch ID and username, and then click **OK**.
Batch ID has a 26-character limit. Use only numbers, letters, underscores (_), or dashes (-). For example: 2025-10-16_Batch3.
- 3 Click **New Batch**.

- 4 After initiation, click **OK** to begin plasma isolation.
- 5 Perform one of the following steps:
 - To load an existing sample sheet that you previously created, select the sample sheet associated with the batch, and then click **OK**.
 - To proceed without selecting a sample sheet, click **No Sample Sheet**.

For information on creating a sample sheet or setting default values, see the *VeriSeq NIPT Solution v2 Software Guide* (document # 1000000067940).



NOTE

Sample type, singleton or twins, must be accurately recorded for each sample in order to ensure proper data analysis.

If you choose No Sample Sheet, make sure you have set default sample values in the Workflow Manager Service Tools.

- 6 Select the batch size, and then click **OK**.
- 7 Select the number of no template controls (NTCs), and then click **OK**.



NOTE

NTC slots are always the last slots selected. For example, with two NTCs in a 24 sample run, positions 23 and 24 are NTCs.

- 8 Confirm that all barcodes are affixed, and then load the samples, tips, and plates (barcode facing right) onto the carrier. Click **OK** after each load prompt.

Sample Batch Size	Carrier Type	Track	Item	Site Position
24, 48, 96	Tip	7–12	1000 µl tips	5
			1000 µl tips (96 batch only)	4, 5
	Tube	15	Prepared blood sample tubes 1–24 (for all batch sizes)	1–24
	Tube	16	Prepared blood sample tubes 25–48 (48 and 96 batch size only)	25–48
	Tube	17	Prepared blood sample tubes 49–72 (96 batch size only)	49–72
	Tube	18	Prepared blood sample tubes 73–96 (96 batch size only)	73–96
	Multiflex	19–24	Empty deep-well plate, Final Plasma – barcoded	4
	Multiflex	19–24	Empty deep-well plate, Intermediate Plasma – barcoded	5
	Reagent	47	[Optional] DPBS for no template control	5

- 9 Make sure that the carriers, labware, and reagents are loaded correctly, and then click **OK** on the Pre-Spin Deck Verification screen.
- 10 Observe the ML STAR perform the automated steps.
- 11 When prompted by the Workflow Manager, make sure the ML STAR loading deck is free of any obstructions to allow the ML STAR to unload the carriers.
- 12 Click **Unload** to unload the deck.
- 13 Remove the Intermediate Plasma deep-well plate.
 - a Inspect the plate for consistent volumes in each well (no pipette errors). The expected volume is 1000 µl.
 - b Note any inconsistencies and record them when the Plasma Isolation procedure is complete.
 - c Seal the plate, load with balance, and centrifuge at 5600 × g for 10 minutes with the brake off or on the lowest setting.
- 14 Click **Yes** to proceed to final Plasma Preparation.
- 15 Remove the plate seal and reload the plate onto the carrier.

Sample Batch Size	Carrier Type	Track	Item	Site Position
24, 48, 96	Multiflex	19–24	Intermediate Plasma deep-well plate	5

- 16 Select the **Intermediate Plasma plate has been spun** checkbox, and then click **OK**.
- 17 Observe the ML STAR perform the automated steps.
- 18 When prompted by the Workflow Manager, make sure that the ML STAR loading deck is free of any obstructions to allow the ML STAR to unload the carriers.
- 19 Click **Unload** to unload the deck.
- 20 When prompted by the Workflow Manager, empty the carriers and deck.
- 21 Remove the Final Plasma deep-well plate.
- 22 Inspect the plate for:
 - ▶ Consistent volumes in each well. Expected volume is 900 µl.
 - ▶ Visible cell pellets.
 - ▶ Excessive hemolysis.

If you observe abnormal visible cell pellets or excessive hemolysis, invalidate the affected sample at the end of the Plasma Isolation method or use Batch Manager. For more information about Batch Manager, see *VeriSeq NIPT Solution v2 Software Guide (document # 100000067940)*.
- 23 When prompted by the Workflow Manager, click **OK**.
- 24 Enter comments about affected wells, and then click **OK**.
- 25 Perform one of the following steps.
 - To continue to cfDNA Extraction, click **Yes**.
 - To stop, click **Exit**.

SAFE STOPPING POINT

If you are stopping, seal the Final Plasma plate and store at 2°C to 8°C for up to 7 days.

Extract cfDNA

Preparation

- 1 Visually examine the Extraction and Accessory Boxes to confirm that the kit is not expired.
- 2 Prepare the following reagents. Label the reservoirs tubs and deep-well reservoirs with the name of the reagents.

Item	Storage	Instructions
Final Plasma deep-well plate from <i>Isolate Plasma</i> on page 12.	2°C to 8°C	If previously stored, let stand for 30 minutes to bring to room temperature. Centrifuge at 1000 x g for 20 seconds. Unseal the Final Plasma deep-well plate before use.
Proteinase K	15°C to 30°C	Slowly add 3.75 ml Proteinase Buffer to each reagent vial. <ul style="list-style-type: none"> • Prepare 3 vials for 24 and 48 samples. • Prepare 4 vials for 96 samples. Cap the vial and vortex until resuspended. NOTE: Do not contaminate the rubber stopper. Getting other substances on it will contaminate future samples. Pool prepared reagent from all vials into a reagent tub.
Wash Buffer II	15°C to 30°C	Add 100 ml 100% EtOH to each reagent bottle. <ul style="list-style-type: none"> • Prepare 1 bottle for 24 and 48 samples. • Prepare 2 bottles for 96 samples. Invert to mix. Mark checkbox on bottle.

- 3 Label 1 new full-skirt plate Intermediate, and apply a plate barcode.
- 4 Label 1 new full-skirt plate cfDNA Elution, and apply a plate barcode.
- 5 Label 1 new deep-well plate Extraction Intermediate, and apply a deep-well plate barcode.

- 6 Apply a plate barcode to the DNA Binding plate.
- 7 Prepare a 70% EtOH cleaning solution (70% EtOH, 30% DNase/RNase-free water) for cleaning of the vacuum system.
- 8 Prepare the vacuum system.
 - a Remove the vacuum manifold and clean with 70% EtOH.
 - b Empty the vacuum waste.
 - c Make sure that the ML STAR vacuum system is on.

Avoid cleaning the gasket with EtOH as this can brittle the material.

Procedure

- 1 Click **OK** to start cfDNA Extraction.
- 2 If the VeriSeq NIPT Method is not already open:
 - a Open the AppLauncher, and click **VeriSeq NIPT Method**.
 - b Enter the Batch ID and username, and then click **OK**.
- 3 Load tips onto the tip carriers as follows, and then click **OK**.

Sample Batch Size	Carrier Type	Track	Item	Site Position
24	Tip	1-6	1000 µl tips	1
		7-12	300 µl tips	1
48	Tip	1-6	1000 µl tips	1, 2
		7-12	300 µl tips	1
96	Tip	1-6	1000 µl tips	1, 2, 3, 4
		7-12	300 µl tips	1

- 4 Load counted tips onto the tip carriers as follows.

Sample Batch Size	Carrier Type	Track	Item	Site Position
24, 48, 96	Tip	49-54	1000 µl tips	1
			300 µl tips	2
			50 µl tips	3

- 5 Enter the location of the first and last tips for each tip rack, and then click **OK**.
- 6 Scan the Extraction Box barcodes.
- 7 Enter the user name or reagent preparer initials, and then click **OK**.
- 8 Scan the Accessory Box barcodes.
- 9 Enter the user name or reagent preparer initials, and then click **OK**.
- 10 Confirm that barcodes are affixed.
- 11 Unseal the Final Plasma deep-well plate, and load plates (barcode facing right) onto the plate carrier as follows, and then click **OK**.

Sample Batch Size	Carrier Type	Track	Item	Site Position
24, 48, 96	Multiflex	19-24	New full-skirt plate, Intermediate – barcoded	1
			New full-skirt plate, cfDNA Elution – barcoded	2
			New deep-well plate, Extraction Intermediate – barcoded	4
			Final Plasma deep-well plate – barcoded	5

- 12 Confirm that the DNA Binding plate is barcoded, then click **OK**.

- 13 For partial plate batches, apply a trimmed plate seal over the unused wells (columns 4–12 for 24 sample batches and columns 7–12 for 48 sample batches).
- 14 Load the DNA Binding plate onto the vacuum manifold with the barcode facing right.
- 15 Select the **Are DNA Binding Plate Columns Sealed?** checkbox, and then click **OK**.
- 16 Load reagent tubs onto the reagent carrier as follows, and then click **OK**.

Sample Batch Size	Carrier Type	Track	Item	Site Position
24, 48	Reagent	47	16 ml Elution Buffer	1
			11 ml Proteinase K	2
96	Reagent	47	16 ml Elution Buffer	1
			15 ml Proteinase K	2

- 17 Transfer the specified reagents to the deep-well reservoirs, and then load the reservoirs onto the deep-well carriers as follows.
- 18 Click **OK**.

Sample Batch Size	Carrier Type	Track	Item	Site Position
24, 48	Deep-well	39–44	125 ml Wash Buffer II	1
			125 ml Wash Buffer I	2
			60 ml 100% EtOH	3
			100 ml Lysis Buffer	4
			60 ml DNase/RNase-free water	5
96	Deep-well	39–44	200 ml Wash Buffer II	1
			125 ml Wash Buffer I	2
			100 ml 100% EtOH	3
			100 ml Lysis Buffer	4
			100 ml DNase/RNase-free water	5

- 19 Wait for the automated reagent volume check to complete.
- 20 Confirm that the vacuum waste is not more than half full (empty recommended), and then click **OK**.
- 21 Confirm the placement of all carriers, labware, and reagents, and then click **OK** on the Extraction Deck Verification screen.
- 22 Observe the ML STAR during the automated steps.
- 23 Centrifuge the DNA Binding plate at 5600 × g for 10 minutes.
- 24 After the final vacuum step, remove the DNA Binding plate and clean the bottom surface with 70% EtOH.
- 25 Seal any uncovered wells on the DNA Binding plate and place it on the empty Final Plasma deep-well plate.
- 26 Centrifuge the DNA Binding plate/Final Plasma plate assembly at 5600 × g for 10 minutes with the brake on.
- 27 Click **OK**.
- 28 During DNA Binding plate centrifugation, complete the vacuum cleaning:
 - a Remove vacuum manifold, and then click **OK**.
 - b Wait for the automated waste disposal to complete.
 - c Clean the vacuum manifold and inside the vacuum system with 70% EtOH, and then replace the vacuum manifold.
 - d Select the **Manifold is on Vacuum** checkbox to initiate the elution plate transfer on vacuum manifold, and then click **OK**.

- 29 After centrifugation, unseal the wells containing samples on the DNA Binding plate and place it on top of the cfDNA Elution plate.
The cfDNA Elution plate is on the vacuum manifold.
- 30 Load the DNA Binding plate with the barcode to the right, and then click **OK**.
- 31 Observe the ML STAR during the automated steps.
- 32 After incubation, select the **Plates are assembled as indicated** checkbox to confirm that the DNA Binding/cfDNA Elution plate assembly is on a support base (if required by centrifuge).
- 33 Seal the uncovered wells on the DNA Binding plate.
- 34 Centrifuge at $5600 \times g$ for 2 minutes with the brake on, and then click **OK**.
- 35 Inspect the cfDNA Elution plate for consistent volumes in each well.
The expected volume is approximately $55 \mu\text{l}$.
- 36 Seal and retain the cfDNA Elution plate for library preparation.
- 37 When prompted by the Workflow Manager, make sure that the ML STAR loading deck is free of any obstructions to allow the ML STAR to unload the carriers
- 38 Click **Unload** to unload the deck.
- 39 Unload all carriers and clean the ML STAR deck, and then click **OK**.
- 40 Enter comments about affected wells, and then click **OK**.
- 41 Perform one of the following steps:
 - To continue to Prepare Libraries, click **Yes**.
 - To stop, click **Exit**.

SAFE STOPPING POINT

If you are stopping, seal the cfDNA Elution plate and store at -25°C to -15°C for up to 7 days.

Prepare Libraries

Preparation

- 1 Visually examine the Library Prep and Accessory boxes to confirm that the kits are not expired.
- 2 Prepare the following reagents. Label the reservoir tubs and deep-well reservoirs with the reagent names.

Item	Storage	Instructions
End Repair Mix	-25°C to -15°C	Thaw at room temperature. Vortex to mix.
A-Tailing Mix	-25°C to -15°C	Thaw at room temperature. Vortex to mix, and then centrifuge briefly.
Ligation Mix	-25°C to -15°C	Thaw at room temperature. Vortex to mix, and then centrifuge briefly.
Resuspension Buffer	2°C to 8°C	Vortex to mix. Return to storage after use.
Hybridization Buffer	-25°C to -15°C	Thaw at room temperature. Vortex to mix. Return to storage after use.
VeriSeq NIPT DNA Adapter Plate	-25°C to -15°C	Thaw at room temperature. Vortex to mix. Centrifuge at 1000 × g for 20 seconds. Apply a plate barcode.
Sample Purification Beads	2°C to 8°C	Let stand for 30 minutes to bring to room temperature. Vortex vigorously before each use. Mix by vortexing or inversion until all beads are in suspension and the mixture is homogenous.
80% EtOH	2°C to 8°C	Prepare fresh. Combine 40 ml 100% EtOH and 10 ml DNase/RNase-free water. Invert to mix.
cfDNA Elution Plate from Extract cfDNA on page 14 .	-25°C to -15°C	If previously stored, confirm that the plate was not stored more than 7 days and thaw at room temperature. Vortex at 1500 rpm for 1 minute. Centrifuge at 1000 × g for 20 seconds.

- 3 Label 1 new full-skirt plate Libraries and apply a plate barcode.
- 4 Make sure that the ML STAR thermal control is on.

Dilute Enzymes

- 1 Combine A-Tailing Mix and Resuspension Buffer in a screw cap tube. Vortex to mix, and then centrifuge briefly.

Sample Batch Size	A-Tailing Mix	Resuspension Buffer
24, 48	900 µl	1200 µl
96	1800 µl	2400 µl

- 2 Combine Ligation Mix and Resuspension Buffer in a screw cap tube. Vortex to mix, and then centrifuge briefly.

Sample Batch Size	Ligation Mix	Resuspension Buffer
24, 48	230 µl	1713 µl
96	440 µl	3278 µl

Procedure

- 1 Click **OK** to start Library Preparation. If the VeriSeq NIPT Method is not already open:
 - a Open the AppLauncher, and then click **VeriSeq NIPT Method**.
 - b Enter the Batch ID and user name, and then click **OK**.
- 2 Confirm that the following consumables are prepared as indicated on the Reagent Preparation screen:
 - ▶ A-Tailing Mix, Ligation Mix, and 80% EtOH.
 - ▶ Sample Purification Beads, End Repair Mix, and the VeriSeq NIPT DNA Adapter Plate.
- 3 Select the checkboxes, and then click **OK**.
- 4 Scan the Library Prep Box barcodes.
- 5 Enter the user name or reagent preparer initials, and then click **OK**.
- 6 Scan the Accessory Box barcodes.
- 7 Enter the user name or reagent preparer initials, and then click **OK**.
- 8 Load tips onto the tip carriers as follows, and then click **OK** for each carrier.

Sample Batch Size	Carrier Type	Track	Item	Site Position
24	Tip	1–6	50 µl tips	1
		7–12	300 µl tips	1, 2
48	Tip	1–6	50 µl tips	1, 2
		7–12	300 µl tips	1, 2, 3, 4
96	Tip	1–6	50 µl tips	1, 2, 3, 4
		7–12	300 µl tips	1, 2, 3, 4, 5

- 9 If you stopped the protocol after the cfDNA Extraction procedure, load counted tips onto the tip carriers as follows.

Sample Batch Size	Carrier Type	Track	Item	Site Position
24, 48, 96	Tip	49–54	1000 µl tips	1
			300 µl tips	2
			50 µl tips	3

- 10 Enter the location of the first tip for each tip rack, and then click **OK**.
- 11 Confirm that barcodes are affixed, and load plates (barcode facing right) onto the plate carrier as follows, and then click **OK**.

Sample Batch Size	Carrier Type	Track	Item	Site Position
24, 48, 96	Multiflex	19–24	cfDNA Elution plate – barcoded	1
			DNA Adapter plate – barcoded	2
			New 96-well full-skirt plate, libraries – barcoded	3
			New 96-well full-skirt plates	4, 5

- 12 Load the deep-well carrier as follows, and then click **OK**.

Sample Batch Size	Carrier Type	Track	Item	Site Position
24, 48, 96	Deep-well	39–44	50 ml 80% EtOH in a deep-well reservoir	1
			New 96-well full-skirt plates	2, 3, 4, 5

- 13 Load reagent tubs onto the reagent carrier as follows, and then click **OK**.

Sample Batch Size	Carrier Type	Track	Item	Site Position
24, 48, 96	Reagent	47	2.5 ml End Repair Mix	1
			Prepared A-Tailing Mix (total volume)	2
			Prepared Ligation Mix (total volume)	3
			10 ml Sample Purification Beads	4
			12 ml Hybridization Buffer	5

- 14 Make sure that the carriers, labware, and reagents are loaded as indicated, and then click **OK** on the Library Deck Verification screen.
- 15 Wait for the automated reagent volume check to complete.
- 16 Observe the ML STAR during the automated steps.
- 17 When prompted by the Workflow Manager, make sure that the ML STAR loading deck is free of any obstructions to allow the ML STAR to unload the carriers, and then click **Unload** to unload the deck.
- 18 Inspect the Libraries plate for consistent volumes in each well.



CAUTION

If well volumes are inconsistent, samples can produce incorrect results.

- 19 If storing, seal and retain the Libraries plate.
- 20 Unload the carriers, clean the deck, and then click **OK**.
- 21 Enter comments about affected wells, and then click **OK**.
- 22 Perform one of the following steps:
- ▶ To continue to Quantify Libraries, click **Yes**.
 - ▶ To stop, click **Exit**.
- 23 Unless you are stopping, proceed immediately with quantification.

SAFE STOPPING POINT

If you are stopping, seal the Libraries plate prior to storage. The Libraries plate is stable for up to 7 days from date of preparation at -25°C to -15°C.

Quantify Libraries

Preparation

- 1 Prepare the following reagents:

Item	Storage	Instructions
DNA Quantification Reagent	2°C to 8°C	Protect from light. Thaw at room temperature for 30-150 minutes. (Removing reagent at start of the Prepare Libraries procedure is recommended.) Vortex to mix, and then centrifuge briefly.
DNA Quantification Standard	2°C to 8°C	Vortex to mix, and then centrifuge briefly.
Libraries plate from Prepare Libraries on page 18	-25°C to -15°C	If previously stored, confirm that the plate was not stored for more than 7 days and thaw at room temperature. Vortex to mix. Centrifuge at 1000 × g for 20 seconds.
Resuspension Buffer	2°C to 8°C	Vortex to mix.

- 2 Turn on the fluorometer 10 minutes before use.
- 3 Apply a plate barcode to a new 384-well plate.
- 4 Apply a plate barcode to a new full-skirt plate.

Procedure

- 1 Click **OK** to start quantification.
- 2 If the VeriSeq NIPT Method is not already open:
 - a Open the AppLauncher, and click **VeriSeq NIPT Method**.
 - b Enter the Batch ID and user name, and then click **OK**.
- 3 Scan the Accessory Box barcodes.
- 4 Enter the user name or reagent preparer initials, and then click **OK**.
- 5 Load tips onto the tip carrier as follows, and then click **OK**.

Sample Batch Size	Carrier Type	Track	Item	Site Position
24, 48	Tip	1-6	300 µl tip rack	1
			50 µl tip rack	2
96	Tip	1-6	300 µl tip rack	1
			50 µl tip rack	2, 3

- 6 Confirm that barcodes are affixed, and then, if needed, unseal the Libraries plate.
- 7 Load plates (barcode facing right) onto the Multiflex carrier as follows, and then click **OK**.

Sample Batch Size	Carrier Type	Track	Item	Site Position
24, 48, 96	Multiflex	19-24	New full-skirt plates – barcoded	1
			New 384-well plate – barcoded	2
			Libraries plate – barcoded	3
			New 96-well full-skirt plates	4, 5

- 8 Load reagent tubes without caps into the tube carrier as follows, and then click **OK**.

Sample Batch Size	Carrier Type	Track	Item	Site Position
24, 48, 96	Tube	46	DNA Quantification Standard	1
			DNA Quantification Reagent	2

- 9 Load reagent tubs onto the reagent carrier as follows, and then click **OK**.

Sample Batch Size	Carrier Type	Track	Item	Site Position
24, 48, 96	Reagent	47	New reagent tub (empty)	1
			16 ml Resuspension Buffer	2

- 10 If you stopped the protocol after the Library Preparation procedure, load counted tips onto the tip carriers as follows.

Sample Batch Size	Carrier Type	Track	Item	Site Position
24, 48, 96	Tip	49-54	1000 µl tips	1
			300 µl tips	2
			50 µl tips	3

- 11 Enter the location of the first and last tips for each tip rack, and then click **OK**.
- 12 Make sure that the carriers, labware, and reagents are loaded as indicated, and then click **OK** in the Quant Deck Verification screen.

- 13 Wait for the automated reagent volume check to complete.
- 14 Observe the ML STAR during the automated steps.
- 15 When prompted by the Workflow Manager, make sure that the ML STAR loading deck is free of any obstructions to allow the ML STAR to unload the carriers.
- 16 Click **Unload** to unload the deck.
- 17 Unload the Libraries plate.
 - a Inspect the plate for consistent volumes in each well.
 - b Seal the Libraries plate and store at room temperature until the fluorometric data analysis is complete.
- 18 Unload the remaining 96-well plates and check for consistent volumes in each well.
Gross volume errors can indicate an issue with pipetting steps.
- 19 Unload the 384-well plate and check for liquid in the appropriate wells.
- 20 Seal the plate with a foil seal.
- 21 Centrifuge at 1000 × g for 20 seconds.
- 22 Incubate at room temperature for 10 minutes, protected from light.
- 23 Unload all carriers and clean the ML STAR deck, and then click **OK**.

**NOTE**

Do not discard quantification reagents until data is obtained. You still require the reagents if you need to perform requantification.

- 24 After incubation, remove the foil seal and load the 384-well plate onto the microplate reader. Make sure that A1 is in the top left corner when loading.
- 25 Double click the VeriSeq NIPT template to open it in SoftMax Pro.
- 26 Select **New Experiment** in the Home tab.
- 27 Select **Read**.
- 28 Export the data as XML as follows.
 - a Right click **Plate**, and then select **Rename**.
 - b Scan the barcode of the Quantification plate, and then click **OK**.
 - c In the upper-left corner of the screen, click the plate icon, and then select **Export** from the menu.
 - d Select the **Expt name** checkbox, set the plate date option to raw, set the output format to XML, and then click **OK**.
 - e Set the output file path and name, and then click **Save**.

The Hamilton computer must be able to access the file location. Do not use spaces in the file name or file path.

Analysis

- 1 In the Workflow Manager, on the Scanner Information screen, enter a fluorometer ID.
- 2 Enter comments about the fluorometer run, and then click **OK**.
- 3 Navigate to the XML quantification file that contains the fluorometric data, and then click **OK**.
- 4 Review the standards curve and sample concentration analysis results, and then click **OK**.
- 5 If you must rescan the plate, click **Rescan**.
Samples are time- and light-sensitive. When necessary, perform the Rescan immediately.
- 6 Enter comments about affected wells, and then click **OK**.
- 7 Assess the results and proceed as follows.
 - If the results pass specification, proceed to Pool Libraries. For specifications, see the quantitation QC metrics and boundaries table in the *VeriSeq NIPT Solution v2 Software Guide (document # 1000000067940)*.
 - If the results fail specification, the system aborts the method. Repeat the quantification procedures beginning with *Preparation on page 20*.
- 8 Perform one of the following steps:

- To continue to Pool Libraries, click **Yes**.
- To stop, click **Exit**.

SAFE STOPPING POINT

If you are stopping, seal the Libraries plate prior to storage. The Libraries plate is stable for up to 7 days cumulative storage at -25°C to -15°C.

Pool Libraries

Preparation

- 1 Prepare the following reagents:

Item	Storage	Instructions
Hybridization Buffer	-25°C to -15°C	Thaw at room temperature. Vortex to mix. Return to storage after use.
Libraries plate from <i>Quantification Procedure on page 21</i> .	-25°C to -15°C	If previously stored, thaw at room temperature. Vortex at 1500 rpm for 1 minute. Centrifuge at 1000 × g for 20 seconds.

- 2 Label an empty pooling tube Pool A. For 96 samples, label a second empty pooling tube Pool B.
- 3 Save the following denature program on the thermal cycler with a heated lid.
 - a Choose the preheated lid option and set to 102°C.
 - b Set the reaction volume to 50 µl.
 - c Set the ramp rate to maximum (≥2°C per second).
 - d Incubate at 96°C for 10 minutes, and then 4°C for 5 seconds.
 - e Hold at 4°C.

Procedure

- 1 Place the Libraries plate on the preprogrammed thermal cycler and run the denature program.



NOTE

Do not denature Libraries plate before the quantification has passed QC metrics, as you may want to perform requantification.

- 2 Centrifuge the Libraries plate at 1000 × g for 20 seconds.
- 3 Select **OK** in the Workflow Manager to start pooling libraries.
- 4 If the VeriSeq NIPT Method is not already open:
 - a Open the AppLauncher, and select **VeriSeq NIPT Method**.
 - b Enter the Batch ID and username, and then click **OK**.
- 5 Select the pool concentration, and then click **OK**.
If necessary, adjust the pooling concentration to achieve the target cluster density of 220–260 k/mm².
- 6 If prompted by the Workflow Manager, perform one of the following steps:
 - ▶ To load a sample sheet, select the sample sheet associated with the batch, and then click **Load**.
 - ▶ To use system default values for remaining sample types, sex reporting, or screen type, click **Use Default** for each setting.

For information on creating a sample sheet, see the *VeriSeq NIPT Solution v2 Software Guide (document # 1000000067940)*.



CAUTION

Before selecting the Use Default option, make sure you have set default values in the Workflow Manager Service Tools. Failure to do so can result in an incomplete analysis of samples.

- 7 Select **Start** to begin timing the denaturing plate.
 8 Load tips onto the tip carriers as follows.

Sample Batch Size	Carrier Type	Track	Item	Site Position
24, 48, 96	Tip	7–12	50 µl filter tips	1

- 9 Load the Denatured Library plate (barcode facing right) onto the Multiflex carrier as follows, and then click **OK**.

Sample Batch Size	Carrier Type	Track	Item	Site Position
24, 48, 96	Multiflex	19–24	Denatured Library plate (barcoded)	1

- 10 Load pooling tubes onto the tube carrier as follows, and then click **OK**.

Sample Batch Size	Carrier Type	Track	Item	Site Position
24, 48	Tube	46	New 2 ml tube, Pool A	1
96	Tube	46	New 2 ml tube, Pool A	1
			New 2 ml tube, Pool B	2

- 11 Load reagent tubs onto the reagent carrier as follows, and then click **OK**.

Sample Batch Size	Carrier Type	Track	Item	Site Position
24, 48, 96	Reagent	47	3 ml Hybridization Buffer	1

- 12 Load tips onto the tip carriers as follows.

Sample Batch Size	Carrier Type	Track	Item	Site Position
24, 48, 96	Tip	49–54	1000 µl filter tips	1
			300 µl filter tips	2
			50 µl filter tips	3

- 13 Enter the location of the first and last tips for each tip rack, and then select **OK**.
 14 Make sure that the carriers, labware, and reagents are loaded as indicated, and then select **OK** in the Pooling Deck Verification screen.
 15 Observe the ML STAR during the automated steps.
 16 Enter comments about affected wells, and then click **OK**.
 17 When prompted by the Workflow Manager, make sure that the ML STAR loading deck is free of any obstructions to allow the ML STAR to unload the carriers.
 18 Click **Unload** to unload the deck.
 19 Unload the tube carrier.
 20 Cap each pooling tube, vortex, and then centrifuge briefly.
 21 Click **OK**.
 22 Sequence libraries as soon as possible after pooling. If necessary, seal the Libraries plate and store at -25°C to -15°C for up to 7 days to allow repooling.

SAFE STOPPING POINT

If you are stopping, cap the pooling tubes and store at -25°C to -15°C for up to 7 days.

Prepare Pooled Libraries for Sequencing

Preparation

1 Prepare the following reagents:

Item	Storage	Instructions
Pool tubes	-25°C to -15°C	If previously stored, thaw at room temperature. Vortex briefly. Centrifuge briefly.

2 Prepare the next-generation sequencing system by completing the following fields in the VeriSeq NIPT Module in Local Run Manager (LRM):

- a Run Name
- b Run Description (optional)
- c Pool Barcode

For more information on using the LRM Module, refer to *VeriSeq NIPT Solution v2 Software Guide (document # 1000000067940)*.



CAUTION

The Pool Barcode entered in the LRM module must match the Pool Barcode entered in the Workflow Manager. Incorrect run configurations are rejected by the analysis software and may require resequencing.

The following procedure describes the proper loading of pooled libraries onto a cartridge-based next-generation sequencing instrument.

Procedure

- 1 Add the following consumables to the reagent cartridge, and then pipette to mix.
 - ▶ 900 µl Hybridization Buffer
 - ▶ 450 µl Pool A
- 2 Proceed with sequencing on a next-generation sequencing system.
For sequencing instructions, see the reference guide for your next-generation sequencing instrument. For a NextSeq 550Dx, refer to the NextSeq 550Dx Instrument Reference Guide (document # 1000000009513) or NextSeq 550Dx Instrument Package Insert (document # 1000000043133).
- 3 If necessary, repeat this procedure for Pool B.
 - ▶ To achieve target cluster density range, the library plate can be repooled using a different pooling concentration on the Hamilton. Repooling invalidates the original pool.
 - ▶ Alternatively, the ratio of pool to HT1 (450+900ul) can be modified to achieve target cluster density range.

Next Generation Sequencing

The VeriSeq NIPT Solution v2 can be used with a next-generation sequencer with the following specifications:

- ▶ capable of 2x36 paired-end reads.
- ▶ compatible with index adapters in the VeriSeq NIPT Sample Prep kit.
- ▶ two channel chemistry.
- ▶ automatic production of .BCL files (raw data from sequencing instrument).
- ▶ 400 million paired-end reads per run.
- ▶ Compatible with VeriSeq NIPT Assay Software v2.

The NextSeq 550Dx is compatible with the VeriSeq NIPT Solution v2.

Sequence Data Analysis

After sequencing is complete, sequencing data is automatically sent to the VeriSeq NIPT Assay Software v2 for analysis and report generation. The report includes classifications for each sample in the batch as well as an assessment of all run QC metrics. The analysis process from sequencing completion to final results takes approximately 4 hours for a 48-sample batch. For detailed information on data analysis and the output file, see the *VeriSeq NIPT Solution v2 Software Guide (document # 1000000067940)*.

Interpretation of Results

The VeriSeq NIPT Solution v2 algorithm employs a sophisticated statistical model which combines several different types of information from the collection of paired-end sequenced library fragments. This model is used to detect regions of the genome which are under or overrepresented in the library of each sample. Importantly, this model accounts for whether the degree of under or overrepresentation is quantitatively consistent with an aneuploid event in the fetal genome at the level of fetal fraction estimated for the library.

For all chromosomes, paired-end sequencing data are aligned with the reference genome (HG19). Unique nonduplicated aligned reads are aggregated into 100 kb bins. The corresponding bin counts are adjusted for GC bias and according to previously established region-specific genomic coverage. Using such normalized bin counts, statistical scores are derived for each autosome by comparing the coverage regions that can be affected by aneuploidy with the rest of the autosomes. A log likelihood ratio (LLR) is computed for each sample by taking into account these coverage-based scores and the estimated fetal fraction. The LLR is the probability of a sample being affected given the observed coverage and fetal fraction versus the probability of a sample being unaffected given the same observed coverage. The calculation of this ratio also takes into account the estimated uncertainty in fetal fraction. For subsequent calculations, the natural logarithm of the ratio is used. The Assay Software assesses the LLR for each target chromosome and each sample to provide a determination of aneuploidy.

During batch creation, the user must define the type of a sample (singleton or twin), the screening type (basic or genomewide), and the sex chromosome reporting (Yes, No, and SCA) desired for each sample. Together, these options determine the information reported for each sample.

For all sample types, the screening type determines which autosomal anomalies are reported. For the basic screening type, only whole chromosome trisomy events involving chromosomes 13, 18 and 21 are reported. For the genomewide screening type, whole or partial chromosome deletion or duplication of any autosomal chromosome are reported. The length of the smallest reportable partial chromosome deletion or duplication is 7 Mb.

For singleton samples, you can disable sex chromosome reporting. You can also configure to report sex chromosome aneuploidies either with or without reporting the sex of euploid samples.

For twin samples, if Yes is selected for sex chromosome reporting, the result is limited to reporting the presence or absence of a Y chromosome in the library. Sex chromosome aneuploidy cannot be reported for twin samples.

A result of ANOMALY DETECTED indicates the sample screens positive for one or more anomalies consistent with the selected screen type and sex chromosome reporting option. When an anomaly is detected, the report provides a description of the anomaly in cytogenetic notation.

The VeriSeq NIPT Assay Software v2 uses statistics generated during sequencing to provide a fetal fraction estimation (FFE) for each sample. The FFE is the estimated fetal cfDNA component which is recovered by the assay and reported as a rounded percentage for each sample. The average standard deviation of this estimate across all samples is 1.3%. The FFE is not to be used in isolation to exclude samples when reporting results.

To make chromosomal representation calls, the VeriSeq NIPT Assay Software v2 utilizes the individualized Fetal Aneuploidy Confidence Test (iFACT), a dynamic threshold metric that indicates whether the system has generated sufficient sequencing coverage, given the fetal fraction estimate for each sample. Negative calls are reported only if the sample meets the iFACT threshold. If a sample fails to achieve this threshold, the QC assessment displays FAILED iFACT and the system does not generate a result.

In addition to iFACT, the VeriSeq NIPT Assay Software v2 assesses several other QC metrics during analysis. The additional metrics include assessments of coverage uniformity on reference genomic regions and the distribution of cfDNA fragment lengths. The QC assessment displays either a QC flag or a QC failure for any metrics outside of the acceptable range. In the case of QC failure, the system does not generate a result for the sample. If a sample fails QC, the sample can be reprocessed provided sufficient plasma volume is in the blood collection tube.

The VeriSeq NIPT Solution v2 generates data for use in a final report. It does not generate a final report for the patient. Customers are responsible for the design and contents of the final report to be delivered to the point of care physician. Illumina is not responsible for the accuracy of wording in the customer's final report.

Performance Characteristics

The following data outlined in the clinical performance and analytical performance sections were generated by using the protocols and materials outlined in the Instructions for Use starting with plasma. All sequencing data for this section were generated on a NextSeq 500/550 sequencing system or a NextSeq 550Dx sequencing system with the following configurations:

	NextSeq 500/550	NextSeq 550Dx
On-instrument Software	NextSeq Control Software 4.0	NextSeq Operating Software 1.3
Reagent Kit Version	NextSeq 500/550 High Output v2.5 (75 cycle) Reagent Kit	NextSeq 550Dx High Output v2.5 (75 cycle) Reagent Kit
Sequencing Method	2x36 paired-end sequencing run in high output mode	2x36 paired-end sequencing run in high output mode

Clinical Study

The clinical accuracy of the VeriSeq NIPT Solution v2 was demonstrated by evaluating plasma samples from pregnant women with singleton and twin pregnancies. Samples were obtained from deidentified banked plasma samples that were previously processed from peripheral whole blood specimens. Over 45,000 samples were considered for study inclusion. These samples underwent previous prenatal screening for fetal chromosome aneuploidies and partial deletions and duplications of 7 Mb or greater. All samples from affected pregnancies and a subset of consecutive samples from unaffected pregnancies were eligible for testing if clinical outcomes were available and sample criteria were met. A total of 2,335 samples were in the testing analysis set. From this set, 2,328 samples were from singleton pregnancies and seven samples were from twin pregnancies.

Of these samples, 28 (1.2%, 28/2335) samples failed assay QC on first pass during the analysis of the completed sequencing data:

- 27 iFACT failures (one XO, 26 unaffected)
- One failure for data outside of expected range

Demographics and Pregnancy Characteristics

Maternal age, gestational age, and trimester of pregnancy are summarized in [Table 7](#) for the samples in the genomewide screen, including known mosaic samples.

The demographics were assessed between the basic and genomewide cohorts and showed no statistical difference. Demographics and pregnancy characteristics were similar whether known mosaics were included or excluded.

Table 7 Demographics and Pregnancy Characteristics

Summary Statistic	Genomewide (including known mosaics)
Number of samples	2307*
Maternal age – years	
Mean	35.08
Standard Deviation	4.04
Median	34.95
25th percentile, 75th percentile	32.31, 37.79
Minimum, maximum	20.22, 53.02
Gestational age at blood draw - weeks	
Mean	10.93
Standard Deviation	1.20
Median	10.57
25th percentile, 75th percentile	10.29, 11.14
Minimum, maximum	10.00, 27.86
Trimester of pregnancy – n (%)	
< First (<14 weeks)	2,252 (98%)
Second	54 (2%)
Third (≥ 27 weeks)	1 (0%)

* Final samples presented contained 7 twins.

Clinical Performance

Results as called by the VeriSeq NIPT Solution v2 were compared to the clinical reference standard outcomes. All study samples had clinical reference standard outcomes (clinical truth) related to fetal chromosomal aneuploidy status and partial deletions and duplications of 7 Mb or greater. The clinical reference standard outcome for samples included in this study depended on results of chromosome analysis or a newborn physical exam with an NGS-based NIPT negative screen. Trained study staff performed classification of clinical reference standard data in accordance with the Medical Coding document from the sponsor.

Chromosome analysis methods included karyotyping, fluorescence in situ hybridization (FISH), or comparative genomic hybridization chromosome microarray (CMA). Chromosome analysis was performed on neonatal or infant peripheral blood or saliva, products of conception (POC) samples, amniocytes, chorionic villi, placental tissues, or postnatal umbilical cord blood.

Mosaicism is defined as the presence of two or more cell lines of different chromosomal composition in an individual. The cell lines originate from the same zygote. The type and level of mosaicism varies and is dependent on the timing of mosaic events during embryogenesis and fetal development. Different types of mosaicism appear in prenatal diagnoses depending on the distribution of abnormal versus normal cell lines over cytotrophoblast, mesenchyme, or the fetus.¹⁰ Although mosaicism can be seen with any chromosome, it is more frequent with rare autosomal aneuploidies (RAA).¹¹ Many of the RAAs present in a mosaic state. In the performance evaluation, mosaic cases were included in genomewide analysis, as the purpose of this screen type for this assay is to detect RAA.

Basic Screen Performance

Samples for the basic screen exclude 16 samples with known mosaicism and an additional 48 samples affected with anomalies for the genomewide screen only, such as RAAs or partial autosomal deletions and duplications. A total of 2,243 singleton and twin samples were included in the analysis. All seven twin pregnancies were correctly detected as trisomy 21 (T21) and are not reported in the following table.

Table 8 Sensitivity and Specificity of the VeriSeq NIPT Solution v2 for Detecting Trisomies 21, 18, & 13 in a Basic Screen for Singleton Pregnancies (Excluding Known Mosaics)

	T21	T18	T13
Sensitivity	> 99.9% (130/130)	> 99.9% (41/41)	> 99.9% (26/26)
2-sided 95% CI	97.1%, 100%	91.4%, 100%	87.1%, 100%
Specificity	99.90% (1982/1984)	99.90% (1995/1997)	99.90% (2000/2002)
2-sided 95% CI	99.63%, 99.97%	99.64%, 99.97%	99.64%, 99.97%

Genomewide Screen Performance

Samples for the genomewide screen contained 36 samples with known mosaicism. A total of 2,307 singleton and twin samples were tested. All seven twin pregnancies were correctly detected as having a chromosome 21 anomaly and are not reported in the following tables.

For the genomewide screen, any anomaly includes trisomies, monosomies, and partial deletions or duplications of 7 Mb or greater.

Genomewide Screen Performance for Any Anomaly

Table 9 Sensitivity and Specificity of the VeriSeq NIPT Solution v2 for Detecting Any Anomaly in the Genomewide Screen (Including Known Mosaics)

	Sensitivity	Specificity
Estimate % (n/N)	95.5% (318/333)	99.34% (1954/1967)
2-sided 95% CI	92.7%, 97.3%	98.87%, 99.61%

Genomewide Screen Performance for Any Anomaly on Chromosomes 21, 18, and 13

Table 10 Sensitivity and Specificity of the VeriSeq NIPT Solution v2 for Detecting Any Anomaly on Chromosomes 21, 18, & 13 in the Genomewide Screen (Including Known Mosaics)

	Chr 21	Chr 18	Chr 13
Sensitivity	96.5% (136/141)	95.5% (42/44)	96.3% (26/27)
2-sided 95% CI	92.0%, 98.5%	84.9%, 98.7%	81.7%, 99.3%
Specificity	99.90% (1987/1989)	99.90% (2000/2002)	99.90% (2005/2007)
2-sided 95% CI	99.63%, 99.97%	99.64%, 99.97%	99.64%, 99.97%

Genomewide Screen Performance for Rare Autosomal Aneuploidy

Table 11 Sensitivity and Specificity of the VeriSeq NIPT Solution v2 for Rare Autosomal Aneuploidy (RAA) in the Genomewide Screen (Including Known Mosaics)

	Sensitivity	Specificity
Estimate % (n/N)	96.4% (27/28)	99.80% (2001/2005)
2-sided 95% CI	82.3%, 99.4%	99.49%, 99.92%

Genomewide Screen Performance for Partial Deletions and Duplications

Table 12 Sensitivity and Specificity of the VeriSeq NIPT Solution v2 for Partial Deletions and Duplications of 7 Mb or More in the Genomewide Screen (Including Known Mosaics)

	Sensitivity	Specificity
Estimate % (n/N)	74.1% (20/27)	99.80% (2000/2004)
2-sided 95% CI	55.3%, 86.8%	99.49%, 99.92%

Partial Deletion or Duplication versus Whole Chromosome Aneuploidy Detection

The VeriSeq NIPT Solution v2 features menu options for both a basic screen and a genomewide screen. In the basic screen, an ANOMALY DETECTED result is only reported when a full aneuploidy is detected on chromosomes 21, 18, or 13 and if all quality control metrics are met. In the genomewide screen, the system detects aneuploidy across all autosomes and partial deletion and duplication events of at least 7 Mb.

While using the genomewide screen, the system gives reporting precedence to a partial deletion or duplication event over the whole chromosome call if the size of the partial deletion or duplication covers less than or equal to 75% of the chromosome on which the event is detected. If the partial deletion and duplication region detected is larger than 75% of the size of the chromosome, the event is reported as a full trisomy or monosomy of the entire chromosome. Therefore, substantially large deletions and duplications that are less than 75% of the size of the chromosome can be indicative of a whole chromosome aneuploidy.

In all samples, the LLR score for the whole chromosome classification is available in the supplementary report. The LLR score should be reviewed with respect to the specified cutoff in [Figure 2 on page 38](#) prior to interpreting the result. Chromosome-level LLR scores exceeding the cutoff provide further support for an interpretation consistent with an aneuploidy of the whole chromosome.

In the clinical study, there were two singleton pregnancy samples with substantially large duplications (one on chromosome 21 and one on chromosome 18) that were less than 75% of the relative size of the chromosome (refer to [Table 13](#)). Both events were reported as a partial duplications rather than a full trisomy for that chromosome. The LLR scores for these events were above the cutoff consistent with an affected outcome for a full trisomy. For either a partial duplication or a full trisomy call, the follow-up management for a positive NIPT call is offering the patient confirmation testing via prenatal diagnosis.

Table 13 Examples of Large Duplication Events Identified in the Genomewide Screen

	Clinical Truth	Genomewide System Output	Size of Anomaly (Mb)	% of Chromosome	LLR Scores
Sample 1	Trisomy 21 singleton	Partial duplication on 21	22.50	48.9%	19.43
Sample 2	Trisomy 18 singleton	Partial duplication on 18	47.00	60.2	12.99

Refer to the *VeriSeq NIPT Solution v2 Software Guide (document # 1000000067940)* for additional information on Quality Control metrics used to report aneuploidy results.

Sex Chromosomes

The VeriSeq NIPT Solution v2 sex chromosome results were compared to the clinical reference standard outcome and are summarized in the following table. The percent concordance was calculated for each sex chromosome within each clinical reference standard outcome. Percent concordance was calculated as the number of samples in which the VeriSeq NIPT Solution v2 sex chromosome call matched the clinical reference standard classification, divided by the total number of samples with the same clinical reference standard classification.

Table 14 Percent Concordance for Fetal Sex Classification*

Fetal Sex Classification		Phenotype from the Newborn Physical Exam		Cytogenetic Results							
Detected	Karyotype	Female	Male	XX	XY	XO	XXX	XXY	XYY	Other**	Missing
Anomaly Not Detected	XX	997	0	21	0	2	0	0	0	0	0
Anomaly Not Detected	XY	0	966	0	15	0	0	0	0	0	1
Anomaly Detected	XO	0	0	0	0	19	0	0	1	0	0
Anomaly Detected	XXX	0	0	0	0	0	17	0	0	1	0
Anomaly Detected	XXY	0	0	0	0	0	0	23	0	1	0
Anomaly Detected	XYY	0	0	0	0	0	0	0	11	0	0
Total		997	966	21	15	21	17	23	12	2	1
Percent Concordant		100	100	100	100	90.5	100	100	91.7	Not Applicable	Not Applicable

* Five twin pregnancies were correctly classified as presence of Y. Two pregnancies were correctly classified as no presence of Y.

** Other cytogenetic results were XXXXX and XXYY.

Positive Predictive Value and Negative Predictive Value of the VeriSeq NIPT Solution v2

Positive predictive value (PPV) and negative predictive value (NPV) of the test provide information regarding the ability of the test to inform clinical decisions based on test sensitivity, specificity, and pretest probability that a fetus is trisomy affected (prevalence). Because PPV and NPV depend on prevalence and the prevalence for these aneuploidies can vary across different subject populations, PPV and NPV were calculated for a range of plausible prevalence values based on the sensitivity and specificity values observed in the basic screen (without known mosaics) of the clinical accuracy study. Table 18 is based on the genomewide screen (with known mosaics).

Table 15 Trisomy 21 Prevalence, PPV, and NPV in Basic Screen (Excluding Known Mosaics)

Prevalence (%)	PPV (%)	NPV (%)
0.05	33.17	> 99.99
0.10	49.82	> 99.99
0.20	66.53	> 99.99
0.50	83.29	> 99.99
1.00	90.93	> 99.99
1.50	93.79	> 99.99
2.00	95.29	> 99.99

Table 16 Trisomy 18 Prevalence, PPV, and NPV in Basic Screen (Excluding Known Mosaics)

Prevalence (%)	PPV (%)	NPV (%)
0.03	23.06	> 99.99
0.05	33.31	> 99.99
0.10	49.99	> 99.99
0.20	66.68	> 99.99
0.30	75.03	> 99.99
0.40	80.04	> 99.99
0.50	83.38	> 99.99

Table 17 Trisomy 13 Prevalence, PPV, and NPV in Basic Screen (Excluding Known Mosaics)

Prevalence (%)	PPV (%)	NPV (%)
0.01	9.10	> 99.99
0.02	16.68	> 99.99
0.05	33.37	> 99.99
0.10	50.05	> 99.99
0.20	66.73	> 99.99


Table 18 Any Anomaly Prevalence, PPV, and NPV in Genomewide Screen (Including Known Mosaics)

Prevalence (%)	PPV (%)	NPV (%)
0.01	1.42	> 99.99
0.02	2.81	> 99.99
0.05	6.74	> 99.99
0.10	12.64	> 99.99
0.20	22.45	99.99
0.50	42.07	99.98
1.00	59.34	99.95
1.50	68.75	99.93
2.00	74.68	99.91

Fetal Fraction Distribution

The distribution of VeriSeq NIPT Solution v2 Fetal Fraction (FF) estimates from the genomewide screen with mosaics are shown by Clinical Reference Standard outcome category in [Figure 1](#).

Figure 1 Fetal Fraction Distribution

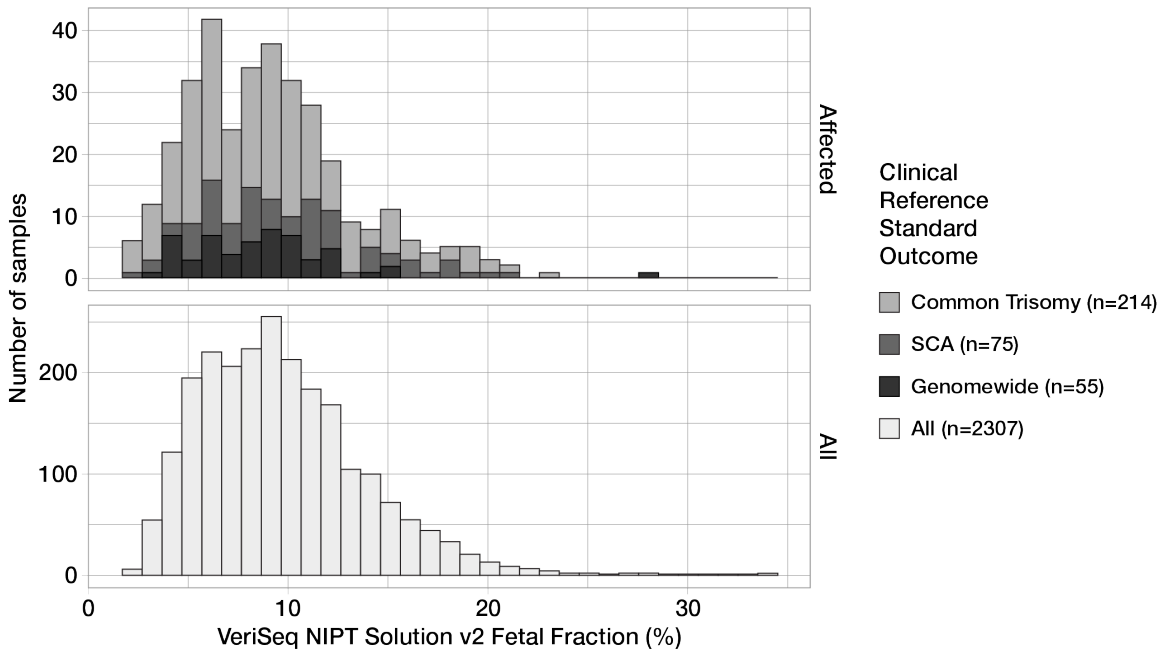

5 samples had anomalies across multiple categories.
 Common trisomy includes samples with trisomy 21, 18, and/or 13.
 Genomewide includes samples with RAA or partial deletions and/or duplications.

FF estimates ranged from 2% to 34% overall with a median of 9% and interquartile (IQ) range of 6% to 12%. The median FF estimate for common trisomies and events detected by the genomewide screen is 8% and for SCAs is 9%. The range in FF estimates was consistent for all the outcomes. There is no apparent shift in the distribution of FF among common trisomies, SCAs, events detected by the genomewide screen, or all samples in the genomewide analysis.

Performance in Twin Pregnancies

Estimating Trisomy 13, 18, and 21 and Chromosome Y Performance in Twin Pregnancies

Due to the low prevalence of trisomy 21, 18, and 13 in twin pregnancies, only a small number of affected twin samples were available for the clinical study. To estimate the performance of the VeriSeq NIPT Solution v2 in twin pregnancies, *in silico* models based on observations from clinical samples were used to simulate populations of twin pregnancies. This simulation was consistent with the intended use population. The distribution of fetal fraction was determined from approximately 4,500 twin samples and compared to the distribution from approximately 120,000 singleton samples. The distribution of fetal fraction conditional on aneuploidy status was determined from singleton putative calls (1,044 trisomy 21, 307 trisomy 18, and 192 trisomy 13). Combining the two distributions allowed for inferences of aneuploidy detection in twins. Sets of dizygotic and monozygotic twins were simulated, and a weighted average representing their prevalence in the intended use population was taken (2 dizygotic: 1 monozygotic) to estimate sensitivity. For specificity, sets of unaffected twins were simulated.

The fraction of each simulated sample affected by the trisomy (ie, the affected fraction) was calculated differently for each sample category:

- ▶ For monozygotic twins, the affected fraction of each sample was set to 1.0 because, in this situation, the trisomy affects both twins.

- ▶ For dizygotic twins, it was assumed that only one twin was affected (to have both dizygotic twins be affected is extremely rare). Affected fraction values were simulated using the known distribution of fetal fraction ratios as determined from sex discordant clinical twin samples. A conservative approach was taken whereby it was assumed that the affected twin always had the lowest fetal fraction of the two twins. A correction factor was applied for fetal fractions being on average lower in trisomy 13 and 18 pregnancies.

- ▶ For unaffected twins, the affected fraction of each sample was set to zero.

For twins affected by either trisomy 18 or 13, the fetal fraction corresponding to the affected fraction of the sample was reduced. The reduction was proportional to the average reduction in fetal fraction observed in clinical data in trisomy 18 or 13 singletons versus euploid singletons.

Both the overall fetal fraction and the affected fraction of each simulated sample were then used to calculate an aneuploidy score using the standard VeriSeq NIPT Solution v2 algorithm. Sensitivity was calculated by determining how often the aneuploidy scores for the simulated affected twins were above the corresponding aneuploidy cutoff. Correspondingly, specificity was calculated by determining how often the aneuploidy scores for the simulated unaffected twins were below the corresponding aneuploidy cutoff (Table 19). 95% confidence intervals were estimated based on the number of real clinical twin samples in the original data set, which were classified as either affected or unaffected by the relevant trisomy.

To estimate chromosome Y sensitivity in twin samples, sets of XY/XY and XX/XY twins were simulated. A weighted average representing their prevalence in the intended use population was taken (1 XY/XY: 1 XX/XY). To estimate chromosome Y specificity in twins, a set of XX/XX twins was simulated. The overall fetal fraction values were simulated according to the known distribution of fetal fraction in clinical twin samples.

For XY/XY and XX/XY twins, corresponding chromosome Y scores were estimated using the known relationship between fetal fraction and chromosome Y scores in clinical singleton samples classified as male. For XX/XY twins only, affected (ie, male) fetal fraction values were simulated using the known distribution of fetal fraction ratios observed between twins from the same pregnancy, as determined from sex discordant clinical twin samples. A conservative approach was taken whereby the affected fraction was selected such that it corresponded to the smaller of the two twins. For each simulated XX/XY sample, the chromosome Y score was multiplied by the affected fraction.

For XX/XX twins, chromosome Y scores were sampled from those scores observed in clinical singleton samples classified as female. The chromosome Y score and the overall fetal fraction were then used to classify each simulated sample as chromosome Y present or chromosome Y absent using the standard VeriSeq NIPT Solution v2 algorithm.

Sensitivity was calculated by determining how often the simulated XY/XY or XX/XY twins were correctly classified as chromosome Y present. Specificity was calculated by determining how often the simulated XX/XX twins were correctly classified as chromosome Y absent. 95% confidence intervals were estimated based on the number of real clinical twin samples in the original data set that were classified as either chromosome Y present or chromosome Y absent.

Table 19 Estimates for Trisomy 21, 18, and 13 in Simulated Population of Twin Pregnancies

	Trisomy 21	Trisomy 18	Trisomy 13	Presence of Y
Sensitivity	96.4%	95.7%	93.6%	> 99.9%
2-Sided 95% CI	(86.4%, 98.9%)	(68.3%, 99.4%)	(64.1%, 98.9%)	(99.9%, > 99.9%)
Specificity	99.9%	> 99.9%	> 99.9%	> 99.9%
2-Sided 95% CI	(99.8%, > 99.9%)	(99.9%, > 99.9%)	(99.9%, > 99.9%)	(99.7%, > 99.9%)

Table 19 provides point estimates and estimated 95% confidence intervals for the sensitivity and specificity of VeriSeq NIPT Solution v2 to detect trisomy 21, 18, 13, and the presence of Y in a simulated population of twin pregnancies consistent with the intended use population. Confidence intervals were estimated based on the

number of QC passing clinical twin samples classified as either affected or unaffected by the relevant trisomy. The sensitivity calculation assumes that two thirds of affected twin pregnancies are dizygotic with one affected twin, while one third of affected twin pregnancies are monozygotic with both twins affected.

The estimates listed in [Table 19](#) pertain to twin pregnancies only. Due to even lower prevalence, data for higher-order pregnancies (triplets or higher) were insufficient to establish appropriate statistical models to estimate accuracy of aneuploidy detection.

Analytical Performance

Precision

To assess and quantify assay precision, a reanalysis of data using the VeriSeq NIPT Solution v2 analysis pipeline software from two previous studies from the VeriSeq NIPT Solution was conducted:

- ▶ Multisite Reproducibility study that comprised three runs by three operators across three sites using a single lot of reagent for a total of nine runs.
- ▶ Within-Lab Precision study that comprised 12 runs at a single site using two ML STARs, two sequencing instrument systems, and three sequencing reagent lots.

The objective of the precision study was to quantify the precision of the assay with regards to trisomy 21 (T21) and Chromosome Y and estimate variability among different instruments, library preparation kits, and sequencing reagent lots.

A 5% fetal fraction T21 pool was created by combining cfDNA extracted from maternal plasma from pregnant women (with a T21 affected fetus) and cfDNA extracted from plasma from nonpregnant women. A 10% fetal fraction maternal-male (XY fetus) cfDNA pool was also created. The sample panel for each study for each run included 4 replicates of the 5% fetal fraction T21 affected sample pool and 20 replicates of the 10% fetal fraction maternal-male cfDNA pool. Testing was performed over 10 days for a total of 21 runs for the two studies combined.

T21 and the presence of chromosome Y were chosen for evaluation based on representativeness of clinical conditions and complexity of anomaly detection. As the second smallest human autosome, the size of chromosome 21 has a direct impact on the sensitivity of T21 detection, particularly at low fetal fraction values such as those used in this study. Chromosome Y, as present in maternal plasma, is exclusively fetal in origin and therefore easier for the assay to detect.

The observed mean and standard deviations for the Chromosome 21 LLR Score and the Chromosome Y normalized chromosomal values (NCV) showed that replicate standard deviation (SD) was the largest source of variability. Variation between sites, instruments, and reagent lots added an insignificant amount of variability, as evidenced by the difference between Total SD and Replicate SD in [Table 20](#) and [Table 21](#).

Table 20 Summary of Multisite (Reproducibility) Sequencing Response Standard Deviation (SD)

Response	N	Mean	Replicate SD	Total Reproducibility SD*
Chromosome 21 LLR Score	36	34.43	11.36	11.36
Chromosome Y NCV	180	190.56	7.96	10.20

* Total includes variability due to site, operator, run, day, and replicate.

Table 21 Summary of Within Lab Sequencing Response Precision

Response	N	Mean	Replicate SD	Total Within Lab SD*
Chromosome 21 LLR Score	48	36.01	9.07	10.25
Chromosome Y NCV	240	198.68	7.63	7.82

* Total includes variability due to sequencing instrument, reagent lot, operator, run, day, and replicate.

An additional study was performed to compare VeriSeq NIPT Solution v2 sequencing precision (total standard deviation) using the version 2.0 of a flow cell versus version 2.5. The study included two types of flow cells (v2.0 and v2.5), three sequencing kit lots, four instrument systems, and two sequencing runs per combination for a total of 48 runs at a single site. One sequencing pool was prepared from cfDNA plates that were manually prepared. The sample panel included 4 replicates of the 5% fetal fraction T21 affected sample pool and 20 replicates of the 10% fetal fraction maternal-male (XY fetus) cfDNA pool. Results from the study are presented in [Table 22](#) and support that there is no difference in sequencing precision when using flow cell v2.0 versus flow cell v2.5.

Table 22 Summary of Flow Cell v2.0 versus Flow Cell v2.5 Sequencing Response Precision

Response	Number of Observations per Version	v2.0 Total SD*	v2.5 Total SD*	Statistical Result**
Chromosome 21 LLR Score	96	9.56	8.44	Statistically Equivalent (p-value=0.25)
Chromosome Y NCV	480	7.74	7.38	Statistically Equivalent (p-value=0.38)

* Total includes variability due to sequencing instrument, reagent lot, run, day, replicate

**Based on F-test for equality of variances (standard deviations squared)

Cross-Contamination

Cross-contamination was assessed in the VeriSeq NIPT Solution sample preparation workflow. Plasma pools from non-pregnant females (XX) and adult males (XY) were tested in a checkerboard pattern in the 96-well plate format across 4 plates. N = 48 each for female and male samples per plate, for a total of 192 female and 192 male samples. None of the female samples demonstrated chromosome Y coverage that was statistically higher than the estimated background, indicating no cross-contamination from male samples within the same plate. No detectable cross-contamination was observed in the VeriSeq NIPT Solution.

Potentially Interfering Substances

The impact of potentially interfering substances was assessed in the VeriSeq NIPT Solution by evaluating the performance of the assay in the presence of such substances.

Albumin, bilirubin, hemoglobin, and triglycerides (endogenous) were each spiked into maternal plasma pools from unaffected female (XX fetus) pregnancies. They were tested at two concentrations for each test substance (n=16 for each). No interference in the performance of the assay was observed.

Table 23 Potentially Interfering Substances (endogenous)

Test Substance	Low Test Concentration (mg/mL)	High Test Concentration (mg/mL)
Albumin	35	50
Bilirubin	0.01	0.15
Hemoglobin	100	200
Triglyceride	1.5	5

Naturally occurring maternal genomic DNA (gDNA) in the plasma can also potentially interfere with assay performance, as it can be extracted along with the fetal cfDNA. Genomic DNA levels at 1.6, 3.3, and 4.9 ng per sample (corresponding to 1, 2, and 3 standard deviations above mean expected gDNA concentration after 7 days of whole blood storage¹²) was added to cfDNA extracted from maternal plasma from unaffected female (XX fetus) pregnancies. Samples were then tested in the VeriSeq NIPT Solution (n=16 for each concentration). No interference in the performance of the assay was observed in the presence of elevated levels of gDNA.

Twenty drug-based potentially interfering substances (exogenous) commonly used or prescribed during pregnancy were tested per EP7-A2 (Interference Testing in Clinical Chemistry; Approved Guideline-Second

Edition). The 20 potential interferents were combined into four pools, spiked into maternal plasma from unaffected female (XX fetus) pregnancies, and tested in the VeriSeq NIPT Solution (N=16 for each pool). No interference in the performance of the assay was observed in the presence of these exogenous substances.

Table 24 Potentially Interfering Substances (exogenous)

Pool 1	Pool 2	Pool 3	Pool 4
Acetaminophen	Diphenhydramine	Albuterol	Cetirizine
Acetylcysteine	Erythromycin	Bupropion	Dextromethorphan
Bisoprolol	Guaifenesin	Caffeine	L-Ascorbic acid
Citalopram	Heparin	Sertraline	Metoprolol
Desloratadine	Lidocaine	Sodium fluoride	Nadolol

Limit of Detection

The Limit of Detection (LOD) is defined as the level of fetal fraction that corresponds to the 95% detection probability of a condition of interest, such as T21. To assess the LOD of VeriSeq NIPT Solution v2 for various common conditions, studies and statistical analyses were conducted.

The probability of detection of a condition of interest in an affected sample processed by VeriSeq NIPT Solution v2 primarily depends on three factors:

- ▶ fetal fraction
- ▶ sequencing depth
- ▶ size and complexity of the genomic region of interest.

Assuming constant sequencing depth, a given aberration is easier to detect in a sample with a higher fetal fraction percentage than in a sample with a lower fetal fraction percentage. Conversely, assuming constant fetal fraction, a given aberration is easier to detect in a sample with a higher sequencing depth than in a sample with a lower sequencing depth. Lastly, aberrations in smaller or more complex genomic regions are harder to detect than aberrations in larger or less complex genomic regions, assuming constant fetal fraction and sequencing depth.


To determine the LOD for T21 detection, samples comprising mixtures of pooled T21 samples and pooled unaffected samples were analyzed. The two types of analyte were mixed over a titration series to create a set of seven fetal fraction levels (0, 2, 3, 4, 5, 6, and 10%). Each level was represented by a total of 10 replicates.

To further increase the resolution of the fetal fraction grid for the LOD analysis, the data from this study was augmented with data obtained from an in silico dilution. The effects of experimental dilution and titration were simulated by the controlled mixing of sequencing data. The data from this in silico titration covered a set of 14 fetal fraction levels (1.25, 1.50, 1.75, 2.00, 2.25, 2.50, 2.75, 3.00, 3.25, 3.50, 3.75, 4.00, 4.25, and 4.50%) with 32 replicates for each level. A probit analysis was applied to the resulting data to determine the LOD for T21.

Independently, a statistical model using fetal fraction, sequencing depth, and genomic size/complexity was developed to predict the detection probability of any aberration in any sample. This model was established from the data corresponding to a set of 1,405 XY samples. The LOD for T21, as predicted by this model, was determined to be concordant with the probit-based estimate described above. This statistical model was used to estimate LOD values for aneuploidies on all autosomes and for partial deletions and duplications.

Figure 2 shows the 95% detection probability for average regions by size and the autosomal limits of detection for all trisomies and all monosomies.

Figure 2 95% Detection Probabilities for Average Regions by Size for the VeriSeq NIPT Solution v2

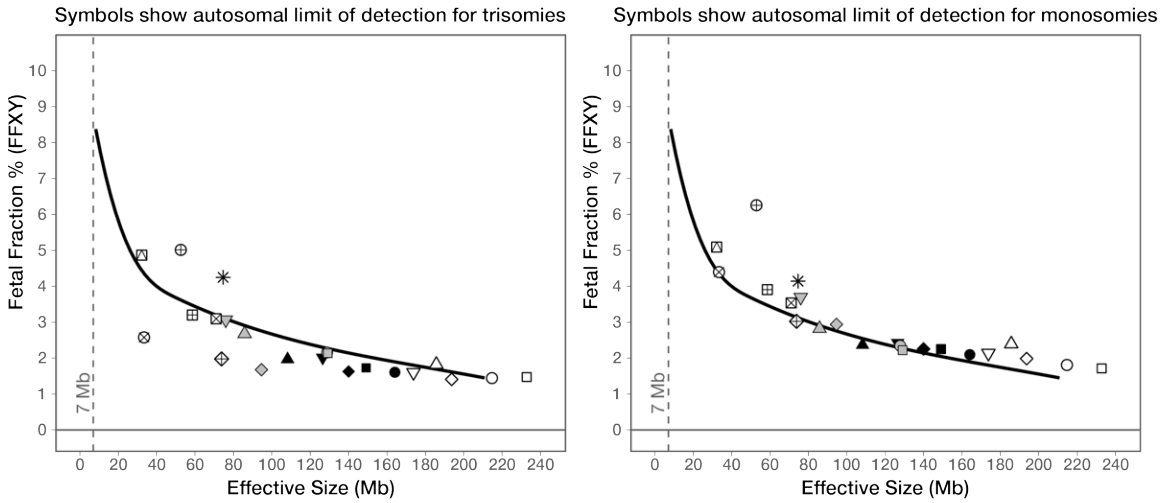

Chr	Symbol	Trisomy		Monosomy	
		LLR Cutoff	LoD (%)	LLR Cutoff	LoD (%)
1	○	7	1.44	13.2	1.80
2	□	9	1.47	13.6	1.71
3	◇	5	1.41	13.8	1.99
4	△	7	1.82	15.2	2.39
5	▽	7.6	1.60	17	2.14
6	●	7.3	1.60	15.4	2.09
7	■	6.6	1.73	14	2.25
8	◆	5.8	1.63	14.8	2.25
9	▲	8	1.97	13.6	2.37
10	▼	8.8	2.01	14.7	2.42
11	⊙	12.2	2.14	15.7	2.35

Chr	Symbol	Trisomy		Monosomy	
		LLR Cutoff	LoD (%)	LLR Cutoff	LoD (%)
12	▣	11.6	2.14	12.8	2.22
13	◇	3	1.68	16.5	2.94
14	△	12.7	2.68	14.7	2.82
15	▽	9.8	3.07	16.4	3.69
16	⊠	10.7	3.10	15.3	3.54
17	*	16.8	4.25	15.7	4.14
18	⊕	3	1.98	11.3	3.02
19	⊗	15.5	5.01	27.5	6.26
20	⊞	10.6	3.20	18.2	3.91
21	⊗	2.5	2.58	13.2	4.40
22	⊞	13.5	4.87	15.3	5.09

Troubleshooting

VeriSeq NIPT Solution v2 Troubleshooting

Failure Mode	Possible Result	Interpretation	Recommended Action	Comments
Insufficient input plasma	Sample QC failure	Insufficient plasma volume	Redraw	Based on visual inspection of plasma volume.
Blood tube failure	No separation of blood into layers	Sample was not centrifuged	Make sure that the centrifuge started and the tube was spun at the correct force. Redraw sample.	
		Improper sample storage or transport (hemolysis of sample)	Redraw sample.	Frozen samples will not separate. Improper transport or storage conditions might lead to hemolysis of samples.

Failure Mode	Possible Result	Interpretation	Recommended Action	Comments
Sample clog / Slow flow	Plasma contamination	Individual samples may clog the binding plate if there is significant contamination in the plasma sample	Inspect sample, if remaining plasma in tube is red or milky, cancel sample and request redraw. If sample appears normal, retest sample.	
	Hardware malfunction	Inadequate digestion of material during extraction	Retest sample. If problem persists in well location with other samples, contact Illumina Technical Support.	
Quantification QC failure	Failed quantification run – Batch median below minimum	Insufficient process yield	Repeat quantification. If repeat also fails, contact Illumina Technical Support.	Passing standards curve metrics indicates issue with library preparation.
	Failed quantification run	Standard curve failure	Repeat quantification. If repeat also fails, contact Illumina Technical Support.	Common causes of standard curve failure include inadequately thawed quantification reagents, inconsistent volumes in wells due to spillage, and DNA quantification reagent degradation (for example, due to light exposure).
Pooling Failure	Failure to complete sample pooling	Pooling Analysis is unable to calculate proper pool volumes	Reevaluate target pool concentration, rerun pooling analysis.	Can occur when all samples in a batch have low quantification values while you have set a high pool concentration (usually higher than 3 to 5 pm).
Individual Sample Analysis QC failure	Sequencing QC failure	Insufficient genetic input OR Mistransfer during sample handling OR Sequencing reagent failure	Check Sample Annotation. Check for similar performance on previous samples in relative plate position. Retest sample.	Indicates either a bad sample input, or a mistransfer on the ML STAR. Insufficient genetic material can come from insufficient cell-free DNA in the plasma or cell-based DNA causing over-dilution of the sample for sequencing.
	Low FF or Non-Excluded Sites (NES) count	Insufficient data generated to make accurate reporting	Retest from plasma.	

VeriSeq NIPT Microlab STAR Troubleshooting

Process Step	Error Code	Error Dialog	Description	User Resolution
Batch Creation	EM0044	The Batch ID entered contains forbidden characters.	The VeriSeq NIPT Solution v2 only accepts numbers, letters, underscores, and dashes for all data fields.	Rename batch using a name that does not contain any special text characters.
Batch Creation	EM0051	The Batch ID is greater than 26 characters in length.	The VeriSeq NIPT Solution v2 limits the length of batch names to 26 characters or less.	Rename batch using a name that is fewer than 26 characters.
Batch Creation	EM0076	Unable to connect to VeriSeq Onsite Server v2	The VeriSeq Onsite Server v2 is not responding to data requests from the Workflow Manager.	Make sure that: <ol style="list-style-type: none"> 1. The ML STAR is connected to the network. 2. The VeriSeq Onsite Server v2 is on. 3. The ML STAR can connect to the VeriSeq Onsite Server v2 (via ping request). 4. If above steps do not resolve the issue, email Illumina Technical Support. 5. Check vacuum waste bottle to see if it is more than half full. If so, empty the waste bottle.
Batch Creation	EM0118	This batch has been failed and cannot be further processed.	The specified batch has already been failed and cannot be processed any further.	The batch record on the VeriSeq Onsite Server v2 indicates that the selected batch has been failed. No further processing is allowed. Create another batch with the desired samples.
Batch Creation	N/A	This batch has already completed processing. Would you like to repool?	The indicated batch has been processed through pooling. The only allowable processing is repool.	To repool, click Re-Pool . OR Abort method and double check Batch Name.
Plasma Isolation	WP0087	Duplicate sample barcodes loaded.	Samples with identical barcodes have been loaded onto the system.	<ol style="list-style-type: none"> 1. Follow the Workflow Manager prompts to identify which samples are duplicates. 2. Remove those samples and either relabel or replace them. 3. Reload samples.
Plasma Isolation	EP0102	Samples specified in the Sample Sheet were not loaded.	Samples included in the sample sheet were not included in the loaded barcodes.	<ol style="list-style-type: none"> 1. Follow the Workflow Manager prompts to identify missing samples. 2. Add missing samples to the batch and reload samples OR Abort method, modify the sample sheet as needed, restart method.
Plate Load	N/A	Venus Barcode Mask Error	The Workflow Manager enforces correct plate-to-batch association using Venus barcode masks.	<ol style="list-style-type: none"> 1. Check plate placement to confirm that the plate layout is correct. 2. Make sure that the plate loaded is the correct plate for the indicated batch.

Process Step	Error Code	Error Dialog	Description	User Resolution
cfDNA Extraction	WE0150	Pressure in the vacuum chamber is too low.	The Workflow Manager will not proceed if the resting vacuum line pressure sense is < 400 Torr.	<ol style="list-style-type: none"> 1. Check for kinks or other obstructions in the vacuum line. 2. Open the waste line release clips, allow for pressure to release, fully close the release line clips. 3. Make sure that the vacuum controller and pump are turned on. 4. If the problem persists, contact IlluminaTechnical Support.
	WE0153	Pressure in the vacuum chamber is too high.	If the measured vacuum pressure is too high prior to starting the pressure control, the system may be malfunctioning.	On the back of the controller, make sure that all vacuum connections and lines are secure.
	WE0996	Vacuum failed to seal.	The system fails to create a vacuum seal on the Binding Plate.	<p>NOTE Do not select OK until the seal failure is completely resolved.</p> <ol style="list-style-type: none"> 1. Make sure that the binding plate is flush against the vacuum manifold. With a gloved hand, forcibly press down on the binding plate. 2. Click OK to continue with cfDNA Extraction. 3. If this error message is displayed more than three times in a run, e-mail Illumina Technical Support.
	WM0219	If Vacuum is on, manually rest the pump.	The vacuum may stay on after a method abort during extraction.	<ol style="list-style-type: none"> 1. On the Vacuum Controller, press the Power button to shut off the vacuum. 2. Wait 10 seconds, and then press the Power button again to turn on the vacuum.
	EE0477	An error has occurred while moving a plate. (iSWAP error)	If an iSWAP error is encountered (plate drop, failed to pick up, and so on) the system will prompt the user to complete the plate move manually.	<p>Make sure that the plate can be recovered (no spilled material).</p> <ul style="list-style-type: none"> - If not, abort run. - If so, follow the displayed instructions to complete the plate transfer manually.
	EE0519	Scanned barcode does not match binding plate barcode on record.	The loaded Binding plate does not match the barcode of the plate removed.	Make sure that the plate being loaded matches the recorded barcode (see the trace log for the expected barcode).

Process Step	Error Code	Error Dialog	Description	User Resolution
API	EA0372	Unable to connect to the data server.	The VeriSeq Onsite Server v2 is not responding to data requests from the Workflow Manager.	Make sure that: 1. The ML STAR is connected to the network. 2. The ML STAR can connect to the VeriSeq Onsite Server v2 (via ping request). 3. The VeriSeq Onsite Server v2 is on.
	EA0774	Connection Error The API server connection failed to validate.	The VeriSeq Onsite Server v2 has stopped responding to data requests from the Workflow Manager.	Make sure that: 1. The ML STAR is connected to the network. 2. The ML STAR can connect to the VeriSeq Onsite Server v2 (via ping request). 3. The VeriSeq Onsite Server v2 is on.
	EA0780	403: Invalid Request The current transaction is not valid.	The data sent violates the system workflow logic.	Consult the error details for more information. Common causes involve inputs that are too long or violate the acceptable character list.

References

- Nagaoka S, Hassold T, Hunt P. Human aneuploidy: mechanisms and new insights into an age-old problem. *Nat Rev Genet.* 2012;13(7):493-504. doi:10.1038/nrg3245.
- Garnder RJ, Sutherland GR, Schaffer LG. *Chromosome Abnormalities and Genetic Counseling.* 4th edition. New York (NY): Oxford University Press; 2012.
- Akolekar R, Beta J, Picciarelli G, Ogilvie C, D'Antonio F. Procedure-related risk of miscarriage following amniocentesis and chorionic villus sampling: a systematic review and meta-analysis. *Ultrasound Obstet Gynecol.* 2015 Jan;45(1):16-26. doi: 10.1002/uog.14636.
- American College of Obstetricians and Gynecologists. Screening for fetal aneuploidy. *Practice Bulletin No. 163.* *Obstet Gynecol.* 2016; 127(5):e123-137.
- Gil MM, Accurti V, Santacruz B, Plana MN, Nicolaides KH. Analysis of cell-free DNA in maternal blood in screening for fetal aneuploidies: updated meta-analysis. *Ultrasound Obstet Gynecol.* 2017 Apr 11. doi: 10.1002/uog.17484.
- Bianchi D, Parker R, Wentworth J et al. DNA Sequencing versus Standard Prenatal Aneuploidy Screening. *N Engl J Med.* 2014;370(9):799-808. doi:10.1056/nejmoa1311037.
- Benn P, Borrell A, Chiu RW, et al. "Position statement from the Chromosome Abnormality Screening Committee on behalf of the Board of the International Society for Prenatal Diagnosis." *Prenat Diagn* 35 (2015): 725-34.
- Gregg AR, Skotko BG, Benkendorf JL, et al. Noninvasive prenatal screening for fetal aneuploidy, 2016 update: a position statement of the American College of Medical Genetics and Genomics. *Genet Med.* 2016: doi:10.1038/gim.2016.97.
- Dondorp W, de Wert G, Bombard Y, et al. Non-invasive prenatal testing for aneuploidy and beyond: challenges of responsible innovation in prenatal screening. *Eur J Hum Genet.* 2015 Nov;23(11):1438-50.
- Grati, et al. "Fetoplacental mosaicism: potential implications for false-positive and false-negative noninvasive prenatal screening results." *Genetics in Medicine* 16 (2014): 620-624.
- Brison, et al. "Predicting fetoplacental chromosomal mosaicism during non-invasive prenatal testing." *Prenat Diagn.* 2018 Mar;38(4):258-266. doi: 10.1002/pd.5223. Epub 2018 Feb 19.
- Norton S, Lechner J, Williams T, Fernando M et al. A Stabilizing Reagent Prevents Cell-free DNA Contamination by Cellular DNA in Plasma During Blood Sample Storage and Shipping as Determined by Digital PCR. *Clin. Lab. Biochem.* 2013;46: 1561-1565. doi: 10.1016/j.clinbiochem.2013.06.002.

- 13 Bianchi D W, et al. "Genome-wide fetal aneuploidy detection by maternal plasma DNA sequencing." *Obstet Gynecol* 119 (2012): 890-901.
- 14 Ehrich M, et al. "Genome-wide cfDNA screening: clinical laboratory experience with the first 10,000 cases." *Genet Med* 19 (2017): 1332-1337.
- 15 Fiorentino F, et al. "The clinical utility of genome-wide cfDNA screening." *Prenat Diagn* 37 (2017): 593-601.
- 16 Pertile, MD, et al. "Rare autosomal trisomies, revealed by maternal plasma DNA sequencing, suggest increased risk of feto-placental disease." *Sci Transl Med* 9 (2017): eaan1240.

Patents and Trademarks

This document and its contents are proprietary to Illumina, Inc. and its affiliates ("Illumina"), and are intended solely for the contractual use of its customer in connection with the use of the product(s) described herein and for no other purpose. This document and its contents shall not be used or distributed for any other purpose and/or otherwise communicated, disclosed, or reproduced in any way whatsoever without the prior written consent of Illumina. Illumina does not convey any license under its patent, trademark, copyright, or common-law rights nor similar rights of any third parties by this document.

The instructions in this document must be strictly and explicitly followed by qualified and properly trained personnel in order to ensure the proper and safe use of the product(s) described herein. All of the contents of this document must be fully read and understood prior to using such product(s).

FAILURE TO COMPLETELY READ AND EXPLICITLY FOLLOW ALL OF THE INSTRUCTIONS CONTAINED HEREIN MAY RESULT IN DAMAGE TO THE PRODUCT(S), INJURY TO PERSONS, INCLUDING TO USERS OR OTHERS, AND DAMAGE TO OTHER PROPERTY, AND WILL VOID ANY WARRANTY APPLICABLE TO THE PRODUCT(S).

ILLUMINA DOES NOT ASSUME ANY LIABILITY ARISING OUT OF THE IMPROPER USE OF THE PRODUCT(S) DESCRIBED HEREIN (INCLUDING PARTS THEREOF OR SOFTWARE).

© 2019 Illumina, Inc. All rights reserved.

All trademarks are the property of Illumina, Inc. or their respective owners. For specific trademark information, see www.illumina.com/company/legal.html.

Contact Information



Illumina
5200 Illumina Way
San Diego, California 92122 U.S.A.
+1.800.809.ILMN (4566)
+1.858.202.4566 (outside North America)
techsupport@illumina.com
www.illumina.com



Illumina Cambridge Limited
Chesterford Research Park, Little Chesterford
Saffron Walden, CB10 1XL
UNITED KINGDOM

Australian Sponsor
Illumina Australia Pty Ltd
1 International Court
Scoresby, Victoria, 3179
Australia

Product Labeling

For a complete reference to symbols that may appear on product packaging and labeling, refer to the symbol key at support.illumina.com on the *Documentation and Literature* tab for your kit.