

Stick the client registration number sticker onto the blood bag. Ensure the needle covers are securely in place, clamps are closed, and one knot is tied in the tubing.




01

Place the blood bag with absorbent cloth between the white gel pack, and then both the blood bag and gel pack into the clear bag. Seal the bag.



02

Place the packaged blood bag into the silver foil bag and ensure that the checklist on the foil bag sticker has been completed.

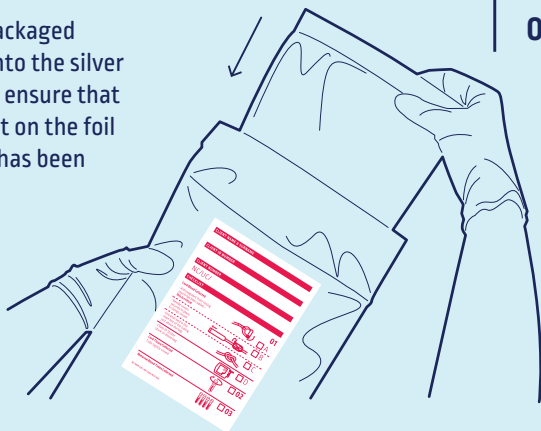

03

If cord tissue has been collected, the date and the time of collection needs to be recorded on the yellow sticker.

Place the bag containing the cord tissue tube in the foil bag along with the packaged blood bag.



04

After the maternal blood test samples have been drawn, place the four filled test tubes in the clear packet marked 'maternal blood collection'.

The date and time of collection needs to be recorded on the red sticker. Place this packet in the silver foil bag.



05

Place the foil bag into the collection kit box.

Please ensure that all information in the important documents folder is completed and placed into the collection kit box.



06

Please ensure that the box is then placed inside the plastic courier bag, and that the waybill is kept on the outside to give to the courier.



07

Please call the Client Services Team using the number below so that they can arrange for the courier to collect the collection kit box.

084 664 4646

Please note the cells do remain viable for 72 hours and we do endeavour to get them to our laboratory within 48 hours.



08