

## Ease of use



Open the outer and inner Omnigen packages.



Pour the contents of the OmniLenz bottle into clean hands.



Orientate the OmniLenz disc correctly in the OmniLenz. Allow 2 minutes rehydration.



Move OmniLenz to your finger ready for insertion.



Sit the patient in an upright seated position to position OmniLenz.



Carefully apply OmniLenz to the ocular surface.

## Comparison of surgical free applications

	OmniLenz®	Amniotic Coated Ring	Bandage Contact Lens
Presentation	Soft lens application of Omnigen	Solid thermoplastic 'conformer' mounted with amniotic membrane	Soft lens only
Available at the point of care	✓	✓	✓
Easy room temperature storage	✓	✗	✓
Direct application from the packet	✓	✗	✓
Easy preparation and application	✓	✗	✓
Patient comfort	✓	✗	✓
Delivery of regenerative factors <sup>1</sup>	✓	✗	✗

1. Allen C.L., et al (2013) An innovative strategy for vision regeneration using a transplantable tissue-engineered corneal construct. PLoS One 8: e7844