

How to use & Orientation Guide

Omnigen®

Distributed through

NEX+BIOLOGICS

How to use:

- Once the inner packaging has been opened Omnigen must be used immediately. Omnigen can be applied without additional preparation and can be placed directly onto the eye without prior rehydration.
- NOTE- Omnigen has its orientation marked on its peripheral edge. At the top of the tissue edge (12 o'clock position) denotes the side with the epithelial surface facing up.
- Omnigen may sometimes have a white patchy appearance on its surface. This is normal and will disappear upon rehydration.


Orientation Guide:

nu Δ vision®
new therapies in sight

Graft


Substrate or permanent inlay

Position graft epithelial side **up**
'gRaft is right'

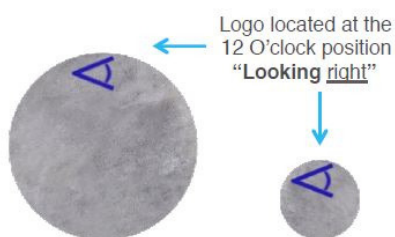

You are looking at the **epithelial** side

'gRaft is right'
(epithelial side up)

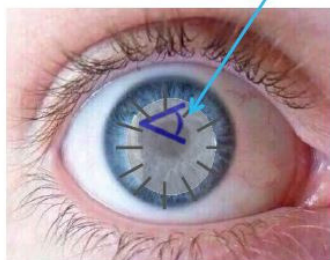


Omnigen®

Orientation guide

Patch


Removable overlay

Position patch epithelial side **down**

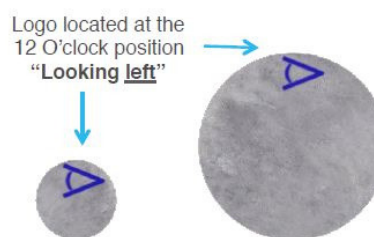

You are looking at the **stromal** side

Looking left
(epithelial side down)

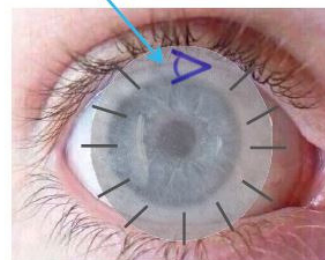

N.B. Always allow 2min rehydration before suturing

It's ready when you are

by NEX+BIOLOGICS

Call +27 [0] 11 697 2900
or visit nextbio.co.za
E: orders@nextbio.co.za

Invest in your future health